



Easy to open – hold the entire box in your hand...



...and gently press down on the lid with the index finger of the other hand.



This makes it easier to open the lid using your thumb.

Keep medication box away from children!



Please read our useful tips and hints prior to use!

- Prior to initial use, please wash/rinse the dosage box! Recommendation – please clean the box with luke-warm water and a mild household dish washing detergent. You can also use the dishwasher (control temperature 70°C), since the dosage boxes are dishwasher-safe.
- It is also recommended to clean the dosage box before refilling it with medication or changing medications!
- Please pay attention to potential special storage conditions for medications such as sensitivity to light and temperature in accordance with the packaging insert from the respective package. In case of uncertainty, please consult with your pharmacist or the manufacturer of the medication. We do not assume liability for the incorrect use of the **ANABOX[®]** or incorrect intake of medications!
- The use of the medication box by persons with mental disabilities should be supervised by a trained person.

Manufacturer:

anmed GmbH

Gartenstraße 13

D-09456 Annaberg-Buchholz

Tel.: +49(0)3733 596300

Fax: +49(0)3733 596301

Email: anmed@t-online.de

www.anabox.info