FR

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AUDISPRAY JUNIOR

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DELICATE CLEANING

AUDISPRAY JUNIOR

AUDISPRAY JUNIOR contains a hypertonic and microbiologically controlled seawater solution, combined with glycerol, which protects the skin of children's ear canals. AUDISPRAY JUNIOR is an alternative to cotton buds.

What is AUDISPRAY JUNIOR for?

AUDISPRAY JUNIOR softens and gently dissolves any excess earwax, thus facilitating its natural removal. AUDISPRAY JUNIOR contains glycerol which protect children's fragile ear with its moisturising and soothing action. AUDISPRAY JUNIOR, with no gas propulsion and fitted with an anti-surge nozzle, is ideal for regular ear cleaning.

Who can use AUDISPRAY JUNIOR?

Children from 3 to 12 years: 1 to 2 sprays per ear twice a week. **AUDISPRAY JUNIOR can be used every day on children that produce a lot of earwax.**

How do I use AUDISPRAY JUNIOR?

1. Point the nozzle towards the child's ear and spray 1 to 2 times, keeping the bottle vertical.
2. Massage the base of the ear for a few seconds.
3. Tilt the child's head to one side to remove excess spray and dry.
4. Rinse the nozzle in hot water after use.

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Contraindications

Do not use on wearers of tympanostomy tubes (grommets), in the event of an ear infection, perforated eardrum and in general on any child who has or has had a medical history (dizziness, medical or surgical procedures, etc.)

Precautions for use

Consult the pharmacist or your doctor in case of doubt or pain before using the product. In the event of an existing wax impaction, AUDISPRAY JUNIOR may lead to the swelling of the impaction, causing discomfort. If this occurs, consult a doctor.

It is recommended to use AUDISPRAY JUNIOR within a year of first using the bottle. Do not use after the expiry date shown on the packaging and the bottle.

Any serious incident connected to use of AUDISPRAY JUNIOR should be reported to the manufacturer and to the competent authority of the Member State where the user lives.

AUDISPRAY JUNIOR is a medical device complying with EU Directive 93/42/EEC

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