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 Dynamic Code AB
Westmansgatan 47
SE-582 16 Linköping
Sweden

Kundekontakt
E-mail:
support@dynamiccode.se
Telefon:
+46 (0)13 465 53 23

Test for Celiac disease (gluten intolerance)

Preparation:


Read carefully...

...through these instructions before you begin. Check that the sterile packages of the test kit are unbroken. If this is not the case, they should not be used. Contact Dynamic Code to receive a new test kit. Note that the blood sample should dry for 3 hours before sending it and that the sample should be sent the same day it is taken. Avoid taking the test the day before the weekend.

This test contains two different sampling kits: one for taking a capillary blood sample and one for taking a buccal sample. **For the test to be accurate, it is necessary for your diet to contain gluten!**

DO AS FOLLOWS: NOTE! Wash and dry your hands before you begin



1. To take the capillary blood test we recommend that you use your middle or ring finger and to not puncture the middle of the fingertip, but a little to the side of the middle.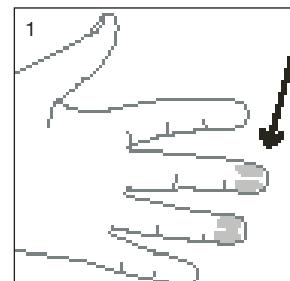

2. Open the alcohol swab packaging.
3. Place the filter paper in front of you.
4. Have a piece of tissue paper or cotton wool at hand to wipe away the first small blood drop.

5. To get a sufficient amount of sample, warm cold hands. You can do this by, for example, rubbing your hands together, swinging your arm around 10 times or placing your hand in lukewarm water for a few minutes.


6. Twist the top of the disposable lancet until it loosens. It can then be removed from the lancet. **It is important not to remove the top without twisting first**, the lancet could then be unusable.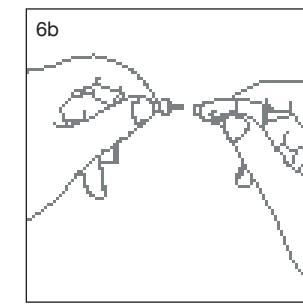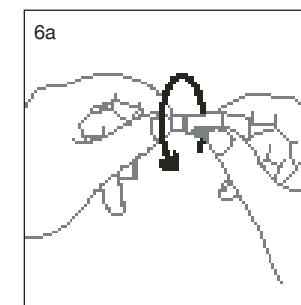


CAPILLARY BLOOD SAMPLE

7. Clean the sampling area by wiping it with the alcohol swab and allow it to air-dry for a few minutes.

8. Place your hand on the tabletop, massage the finger you intend to use for the test and press down hard with your thumb just above the last joint of that finger. The fingertip will become a darker colour as it accumulates blood. Be careful not to touch the area you cleaned when you press your finger with your thumb.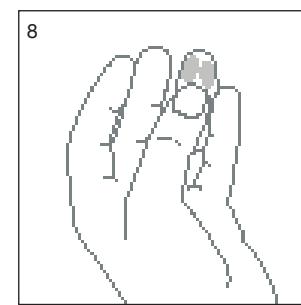

9. Press the lancet to your finger until you feel a needle prick. Release the pressure by removing your thumb from your finger.

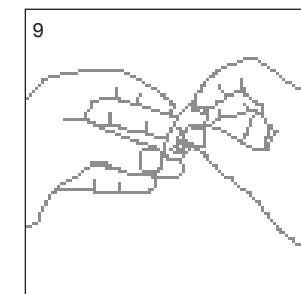
10. **IMPORTANT!** Quickly wipe away the first small drop of blood with tissue paper or cotton wool.

11. Keep the hand in as low a position as possible and carefully press along the whole length of the finger from palm to fingertip. Collect the blood by allowing the blood to drop freely and be absorbed by the filter paper.

12. Try to get as much blood as you can without pressing too hard or for too long. The blood spot should contain at least three blood drops, optimally the blood spot would be 2 cm in diameter. Do not puncture more than once.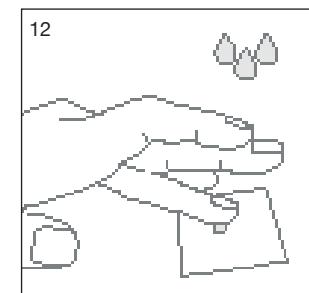

13. Put on a plaster if needed.

14. Carefully remove one label with the analysis code from the leaflet and affix it onto the filter paper. Allow the filter paper to air-dry for three hours.

15. Place the filter paper in the bag and place it in the kit box.

BUCCAL SAMPLE

DO AS FOLLOWS:


1. The buccal sample is taken with the enclosed buccal swab. The swab is in a sterile paper/plastic packaging.

2. Rinse your mouth with water.


3. Place the buccal swab contained in the paper/plastic packaging in front of you. Carefully remove the other label with the analysis code from the leaflet and affix it onto the paper/plastic packaging containing the buccal swab.


4. Open only the top of the paper/plastic packaging. Pull out the buccal swab.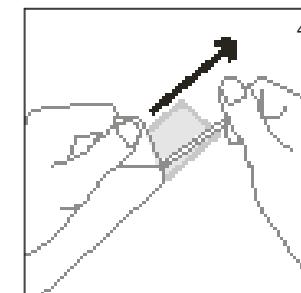

5. Rub the buccal swab on the inside of the cheek.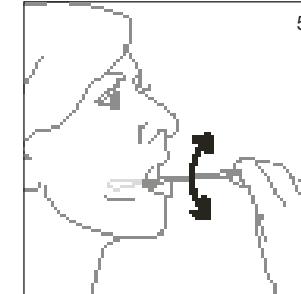

6. Place the buccal swab back in the paper/plastic packaging. Pinch the opening tabs when putting it back in so that the swab slides in easily.

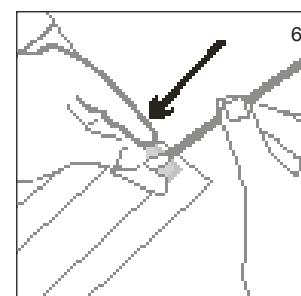7. Place the buccal swab in the kit box and seal it. Tear off the part of the box with the instructions and the suspension tab along its perforated edge. The box serves as the return parcel and is posted to the laboratory by mail service.

**Analysis results can be retrieved via
www.carestore.com/en**

Using your analysis code found on the leaflet.

**IT IS IMPORTANT TO TEAR OFF AND
SAVE YOUR CODE!**

 Dynamic Code AB
Westmansgatan 47
SE-582 16 Linköping
Sweden

Customer service
E-mail:
support@dynamiccode.se
Telephone:
+46 (0)13 465 53 23

S002 rev 1.0