

3M Nexcare™ ColdHot traditional

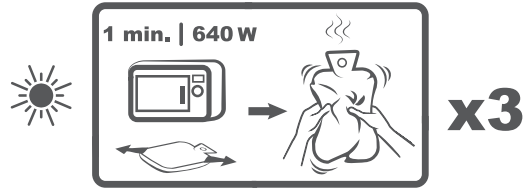ColdHot teddy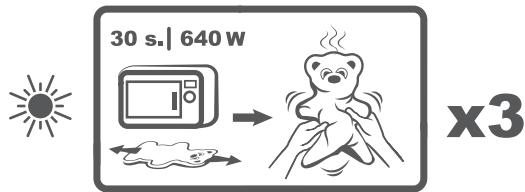

Made in Germany for

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GB Reusable gel warm bottles

The Nexcare™ ColdHot Traditional and Teddy are reusable gel filled warm bottles with a velvety, soft touch fabric finish for immediate and comfortable usage. The product stays warm for several hours providing soothing heat for the body. The gel is non-toxic and safe for the whole family.

Natural therapeutic power of heat

Hot therapy

Heat helps to reduce muscle spasm and to decrease pain, soothes and relaxes.

When to use

- As body warmer or as bed warmer.
- For backache.
- Stomach ache.
- For common cold or flu symptoms.
- Period pain (Traditional).

Apply to the affected area for as long as is comfortable. Re-warm pack once it reaches room temperature.

Preparation - Heat up only in microwave

TRADITIONAL
Lay product flat in microwave set to 640W and heat 3 x 1 min.

TEDDY
Lay product flat in microwave set to 640W and heat 3 x 30 sec.

After each heating knead pack to distribute heat evenly.

Warnings and Precautions:

- For external use only.
- Do not sit or lean on the gel pack or apply excessive pressure as it may cause the pack to rupture and/or leak. Do not use while sleeping.
- Do not use on broken or sensitive skin.
- People with known or suspected nerve or circulatory problems should not use this product unless prescribed by physician.
- Heat may be sufficiently analgesic to produce local burns.
- Supervise use by children or people with cognitive disabilities.
- Do not use ColdHot Teddy as a toy.
- Stop application in case of any discomfort.
- If condition persists, consult your physician.
- Never heat the packs when folded or creased. Leakage may result.
- Check for leaks before use.
- Never use damaged pack. Throw away if leaking.
- Be careful when removing hot pack with bare hands from microwave.
- If pack is too hot, wait for it to cool down before applying to body.
- Packs will be damaged if overheated.
- Do not heat in an oven or grill.
- For unknown microwave oven power, please use the lowest power setting (defrost setting) and warm up the product in 30 seconds steps.

Gel Pack Care Instructions:

The warm bottle can be washed by hand in a mild detergent. Do not scrub. Do not use alcohol or other solvents to clean the pack.

Disposal

Disposal of product in regular household waste.

Explanation of symbols:

- Caution, see instructions for use.
- Manufacturer.
- Not Made With Natural Rubber Latex.
- Not suitable for children under three years.
- LOT Lot Number
- xxxx yy
- Expiry Date: Year-month
- Date of issue of the instruction for use 12.2017

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3M, Nexcare and ColdHot are trademarks of 3M.