
Mepore[®] Film

STERILE | **EO**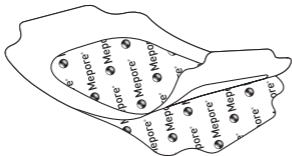

Single use
Usage unique

CE 0086

**MÖLNLYCKE
HEALTH CARE**

USA 1-800-882-4582

Canada 1-800-494-5134

Manufacturer

Mölnlycke Health Care AB

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Mepore® Film

Transparent, breathable, adhesive coated polyurethane film

en

Product Description

Mepore Film is a flexible, transparent, breathable film dressing made of polyurethane, coated with a polyacrylic adhesive. The dressing is supported with a transparent sheet paper frame to ease the application.

Mode of action

Mepore Film provides a flexible, transparent covering to protect the wound whilst conforming to surface irregularities and body contours.

Mepore Film maintains a moist environment yet due to its vapour permeability allows excess moisture pass away from the wound and surrounding skin.

Mepore Film provides a barrier to fluid strike-through and bacterial contamination.

Areas of use

Mepore Film is intended for a wide range of wounds in the granulation phase such as superficial burns, IV sites, abrasions, lacerations, donor sites, superficial pressure ulcers, closed surgical wounds and may help to prevent skin breakdown.

Mepore Film can be used as a conformable fixation device or as a secondary dressing to cover dressings such as hydrogels, alginates and foam dressings.

Method of use

en

Application:

1. Clean the application area. Dry the healthy surrounding skin.
2. Chose the correct dressing size to allow a margin which adheres to dry healthy skin around the wound or catheter site. The dressing may be cut and can be overlapped.
3. For size 10x25 cm (4x10 inches) and 15x20 cm (6x8 inches) only: Remove center cut-out and discard.
4. Remove the protection paper (printed Mepore) to expose the adhesive.
5. Apply Mepore Film and remove the paper frame.
6. Remove the two white paper side tabs.

Dressing change and removal

1. The dressing may be left in place for several days depending on the wound/skin condition or as indicated by good clinical practice.
2. Lift a corner of the dressing and gently pull the film along the skin (in the direction of hair growth).

Precautions

Mepore Film should not be applied on patients who are sensitive to acrylic adhesive.

Mepore Film should not be used directly on full thickness wounds involving muscle, tendon or bone or on third degree burns.

If reused performance of the product may deteriorate, cross contamination may occur.

Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilise.

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CANADA Mölnlycke Health Care Inc.,
2010 Winston Park Drive, Suite 100,
Oakville, Ontario, L6H 5R7
Tel: +1 905 829 1502

CHINA Mölnlycke Health Care
(Shanghai) Co., Ltd.
RM 629, No.8 Hua Jing Road,
Wai Gao Qiao FTZ Shanghai, China
Tel: +86 10 5128 8571

CZECH REPUBLIC Mölnlycke Health Care s.r.o.
Hajkova 2747/22, 130 00 Prague 3
Tel: +420 221 890 517 (**Reception**)
Tel: +420 221 890 511 (**Customer Service**)

DENMARK Mölnlycke Health Care ApS,
Gydevang 33, DK-3450 Allerød
Tel: +45 48 16 82 68
Customer Service Tel: +80 886 810

ESTONIA Mölnlycke Health Care OÜ,
Vabaõhumuuseumitee 4-67, EE-13522 Tallin
Tel: +372 671 1520

FINLAND Mölnlycke Health Care Oy,
Pitäjänmäentie 14, 3 krs, FI-00380 Helsinki
Tel: +358 201 622 300
Customer Service Tel: 0800 113 442

FRANCE Mölnlycke Health Care SAS,
13 Allée du Château Blanc
59290 WASQUEHAL
Tel: +33 320 122 555
Customer Service Tel: 0800 910 292

GERMANY Mölnlycke Health Care GmbH
Grafenberger Allee 297
DE-40237 Düsseldorf
Tel: +49 211 92 08 80
Customer Service Tel: 0800 186 21 80

HUNGARY Mölnlycke Health Care Kft
HU-1134 Budapest, Dévai u. 26-28. III. em.
Tel: +36 1 477 3080

INDIA Mölnlycke Health Care India Pvt Ltd
Universal Business Centre
Second Floor, Universal Trade Tower
Sector 49, Gurgaon Sohna Road
Gurgaon 122 018, Haryana
Tel: +91 124 4696 008

ITALY Mölnlycke Health Care s.r.l.,
Via Marsala 40/C, IT-21013 Gallarate (VA)
Tel: +39 0331 714 011

JAPAN Mölnlycke Health Care KK,
4F Concieria Nishi-Shinjuku Tower's West
6-20-7 Nishi-Shinjuku,
Shinjuku-Ku, Tokyo 160-0023
Tel: +81 3 6914 5004

KOREA Mölnlycke Health Care Korea Co., Ltd
2nd floor, Hansol Richville.
46 Wiryeseong-daero,
Songpa-gu, Seoul, 05627
South Korea
Tel: +82 2 3789 1402

LATVIA Mölnlycke Health Care AB,
Dzelzavas 120 M, Riga LV-1021
Tel: +372 6711 522

LITHUANIA Mölnlycke Health Care AB,
J. Galvydzio g. 3, LT 08236 Vilnius
Tel: +370 5 2274 58 37

NETHERLANDS Mölnlycke Health Care B.V.
Postbus 3196, 4800 DD Breda
Tel: +31 76 521 9663

NORWAY Mölnlycke Health Care AS
Postboks 6229 Etterstad, NO-0603 Oslo
Tel: +47 22 70 63 70
Customer Service Tel: 800 161 78

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Mölnlycke Health Care Ltd.
USA Mölnlycke Health Care US, LLC