Mepore® Film & Pad

STERILE | **EO**

Single use
Usage unique

CE 0086



**MÖLNLYCKE
HEALTH CARE**

USA 1-800-882-4582
Canada 1-800-494-5134

Manufacturer

Mölnlycke Health Care AB
Gamlestadvägen 3C, Box 13080, SE-402 52 Göteborg, Sweden

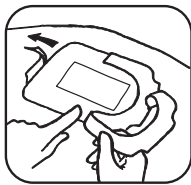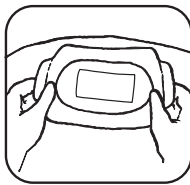MD

- en** Medical Device
- de** Medizinprodukt
- fr** Dispositif médical
- es** Dispositivo médico
- nl** Medisch hulpmiddel
- sv** Medicinsk utrustning
- it** Dispositivo medico
- fi** Lääkinnällinen laite
- pt** Dispositivo médico
- da** Medicinske anordninger
- el** Ιατρική συσκευή
- pl** Urządzenie medyczne
- cs** Lékařský přístroj
- hu** Orvosi eszköz
- no** Medisinsk utstyr
- sl** Medicinska naprava
- bg** Медицинско изделие
- ro** Dispozitiv medical
- sk** Lekársky prístroj
- tr** Tıbbi Cihaz
- lt** Medicinos prietaisas
- lv** Medicīniska ierīce
- et** Meditsiiniseade
- ru** Медицинское изделие
- hr** Medicinski uređaj
- sr** Medicinsko sredstvo

- en** Do not use if package is damaged
- de** Bei beschädigter Verp. verwerfen
- fr** Ne pas utiliser si l'emballage est endommagé
- es** No utilizar si el envase está dañado
- nl** Niet gebruiken indien verpakking beschadigd is
- sv** Skall inte användas om förpackningen är skadad
- it** Non utilizzare se la confezione è danneggiata
- fi** Älä käytä jos pakkaus vaurioitunut
- pt** Não usar se embalagem danificada
- da** Brug ikke, hvis pakken er beskadiget
- el** Μην το χρησιμοποιήσετε αν η συσκευασία είναι κατεστραμμένη
- pl** Nie stosować, jeśli opakowanie jest uszkodzone
- cs** Nepoužívejte, je-li obal poškozen
- hu** Ne használja, ha a csomagolás sérült
- no** Må ikke brukes hvis forpakningen er skadet
- sl** Ne uporabite, če je ovoj poškodovan
- bg** Не използвайте, ако опаковката е увредена
- ro** Nu utilizați dacă ambalajul e deteriorat
- sk** Nepoužívajte, ak je obal poškodený
- tr** Ambalaj hasarlıysa kullanmayın
- lt** Nenaudoti, jei pakuotė pažeista
- lv** Nelietot, ja iepakojums ir bojāts
- et** Kahjustatud pakendi korral mitte kasutada
- ru** Не использ., если упак. поврежд.
- hr** Ne koristiti ako je pakiranje oštećeno
- sr** Ne koristiti ako je omot oštećen

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Absorbent film dressing

Description

The Mepore Film & Pad consists of a transparent polyurethane film coated with a polyacrylic adhesive and a wound pad. The wound pad is an absorbent viscose pad with a low-adherent wound contact layer. The vapour permeable film is water resistant and provides a barrier which protects from fluid strike through and bacterial contamination. Mepore Film & Pad conforms well to body contours.

Intended use

Mepore Film & Pad is intended for use on low to moderately exuding wounds, such as surgical wounds, cuts and abrasions.

Instructions for use

Application:

1. Open the sterile pack and remove the dressing.
2. Clean the application area. Dry the healthy surrounding skin.
3. Remove the protection paper (printed Mepore) to expose the adhesive.
4. Apply Mepore Film & Pad and remove the paper frame.
5. Remove the two white paper side tabs.

Dressing change and removal:

1. The dressing may be left in place for several days depending on the wound/skin condition or as indicated by good clinical practice.
2. Lift a corner of the dressing and gently pull the film along the skin (in the direction of hair growth).

Frequency of change

The frequency of dressing changes should be governed by the condition of the wound in accordance with accepted clinical practice.

Precautions

Mepore Film & Pad should not be applied on patients who are sensitive to acrylic adhesive.

Care should be taken to minimise skin tension that could cause mechanical damage to the skin.

The wound should be inspected for signs of infection according to clinical practice. In case of clinical signs of infection (pain, redness, swelling or unusual odour or discharge) continued use of Mepore Film & Pad should be determined by a health care professional.

Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilise.

If reused performance of the product may deteriorate, cross-contamination may occur.

Presentation

Each dressing is presented sterile, individually wrapped in a peel pouch.

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CANADA Mölnlycke Health Care Inc.,
2010 Winston Park Drive, Suite 100,
Oakville, Ontario, L6H 5R7
Tel: +1 905 829 1502

CHINA Mölnlycke Health Care
(Shanghai) Co., Ltd.
RM 629, No.8 Hua Jing Road,
Wai Gao Qiao FTZ Shanghai, China
Tel: +86 10 5128 8571

CZECH REPUBLIC Mölnlycke Health Care s.r.o.
Hajkova 2747/22, 130 00 Prague 3
Tel: +420 221 890 517 (**Reception**)
Tel: +420 221 890 511 (**Customer Service**)

DENMARK Mölnlycke Health Care ApS,
Gydevang 33, DK-3450 Allerød
Tel: +45 48 16 82 68
Customer Service Tel: +80 886 810

ESTONIA Mölnlycke Health Care OÜ,
Vabaõhumuuseumitee 4-67, EE-13522 Tallin
Tel: +372 671 1520

FINLAND Mölnlycke Health Care Oy,
Pitäjänmäentie 14, 3 krs, FI-00380 Helsinki
Tel: +358 201 622 300
Customer Service Tel: 0800 113 442

FRANCE Mölnlycke Health Care SAS,
13 Allée du Château Blanc
59290 WASQUEHAL
Tel: +33 320 122 555
Customer Service Tel: 0800 910 292

GERMANY Mölnlycke Health Care GmbH
Grafenberger Allee 297
DE-40237 Düsseldorf
Tel: +49 211 92 08 80
Customer Service Tel: 0800 186 21 80

HUNGARY Mölnlycke Health Care Kft
HU-1134 Budapest, Dévai u. 26-28. III. em.
Tel: +36 1 477 3080

INDIA Mölnlycke Health Care India Pvt Ltd
Universal Business Centre
Second Floor, Universal Trade Tower
Sector 49, Gurgaon Sohna Road
Gurgaon 122 018, Haryana
Tel: +91 124 4696 008

ITALY Mölnlycke Health Care s.r.l.,
Via Marsala 40/C, IT-21013 Gallarate (VA)
Tel: +39 0331 714 011

JAPAN Mölnlycke Health Care KK,
4F Concieria Nishi-Shinjuku Tower's West
6-20-7 Nishi-Shinjuku,
Shinjuku-Ku, Tokyo 160-0023
Tel: +81 3 6914 5004

KOREA Mölnlycke Health Care Korea Co., Ltd
2nd floor, Hansol Richville.
46 Wiryeseong-daero,
Songpa-gu, Seoul, 05627
South Korea
Tel: +82 2 3789 1402

LATVIA Mölnlycke Health Care AB,
Dzelzavas 120 M, Riga LV-1021
Tel: +372 6711 522

LITHUANIA Mölnlycke Health Care AB,
J. Galvydzio g. 3, LT 08236 Vilnius
Tel: +370 5 2274 58 37

NETHERLANDS Mölnlycke Health Care B.V.
Postbus 3196, 4800 DD Breda
Tel: +31 76 521 9663

NORWAY Mölnlycke Health Care AS
Postboks 6229 Etterstad, NO-0603 Oslo
Tel: +47 22 70 63 70
Customer Service Tel: 800 161 78

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Mölnlycke Health Care Ltd.
USA Mölnlycke Health Care US, LLC