# Pages

DE - 2	EL - 50
EN - 5	JA - 53
ES - 8	SK - 56
FR - 11	SL - 59
IT - 14	KO - 62
NL - 17	LT - 64
PL - 20	RU - 67
PT - 23	BG - 69
SV - 26	LV - 73
FI - 29	TR - 76
DA - 32	ET - 79
NO - 35	RO - 82
IS - 38	UK - 85
AR - 41	MK - 88
BIH - 44	SR - 91
CS - 47	HR - 94

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Datum: 2015-12

# Instructions for Use

Read the instructions for use before starting the Otovent® treatment.

## **Why use Otovent®?**

Otovent® can be used in situations where you cannot equalize a negative middle ear pressure. This condition is caused by Eustachian tube dysfunction and may follow after an upper respiratory tract infection, an episode of acute otitis media or in connection with e.g. flying, diving or pressure chamber treatment. If left untreated, Eustachian tube dysfunction can lead to secretory otitis media (also known as otitis media with effusion, OME, or glue ear). This condition includes collection of fluid in the middle ear and may lead to a sensation of heaviness and impaired hearing.

## **Who can use Otovent®?**

Otovent® can be used by children aged 3 years and up and by adults. The treatment of children should always be supervised by an adult.

## **How does the Otovent® method work?**

The Otovent® method provides the pressure required to open the Eustachian tube to help equalize the middle ear pressure. The middle ear is thereby ventilated, any effusion can be cleared and symptom relief achieved.

## **How often and how long should Otovent® be used?**

Initially, Otovent® should be used at least three times a day (morning, noon, evening) with 1 inflation in each nostril at each occasion. After one week, it should be used at least twice a day (morning and evening). The normal duration of treatment is 2-3 weeks, after which a physician should decide whether or not to continue the treatment. Each medical balloon may be inflated a maximum of 20 times.

For use in connection with flying Otovent® should be used as soon as the descent begins with 3-4 inflations before landing.

For use after diving or hyperbaric pressure treatment Otovent® can be used several times to relieve the symptoms.

### **When shall Otovent® not be used?**

Otovent® shall not be used during an episode of acute otitis media and/or upper respiratory tract infection.

### **Warnings and Precautions**

 The balloons contain LATEX. If you are allergic to latex or latex products consult your physician before using this product. If you experience any allergic symptoms, such as rash, itching, difficulty in breathing or swelling, stop treatment and consult your physician.

Keep out of reach of children.

The treatment of children should always be supervised by an adult.

Never use toy balloons in performing the Otovent® treatment, as toy balloons do not give the proper inflation pressure and may cause treatment failure.

If you suffer from frequent minor nosebleeds, any manipulation of the nose may provoke nosebleed, such as nose-picking, nose-blowing or balloon inflation. This condition is a nuisance, but usually harmless.

If you experience severe dizziness using Otovent®, stop treatment and consult your physician.

### **How to use Otovent®**

1. Connect the balloon to the flat end of the nose piece.
2. Hold the ball-shaped part of the nose piece firmly against your left nostril with your left hand. Compress your right nostril, using your right index finger.
3. Inhale deeply through your mouth, then close your mouth and inflate the balloon by blowing through your left nostril until the balloon is the size of a grapefruit.

4. Still with the inflated balloon tight to the left nostril, perform some swallowing manoeuvres.
5. Repeat the procedure through your right nostril.

Some patients may experience discomfort in the ear or dizziness during inflation. This initial sensation will decrease during the next inflation and is an indication that the procedure is working correctly.

### **Maintenance**

Clean the nose piece with mild detergent after use. Rinse the nose piece with clean water. Discard used balloons when they have been inflated 20 times.

### **Storage**

When not in use, protect the nose piece and the balloons against light. Keep the items in the box.

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