10 cm/4 in.

Used at hospitals
as Sorbact®
Surgical Dressing

EN Sorbact® Secure Protects and keeps the wound clean. Showerproof dressing that adheres securely, protecting from outside contamination. The unique material binds bacteria from the wound, which facilitates the healing process and prevents infections. **Usage:** Minor wounds, such as cuts, lacerations, abrasions and surgical wounds. **Instructions:** Clean the wound. Remove the white protective film. Apply the dressing to dry skin with the green surface in direct contact with the wound. Press firmly around the edges. Remove the outer film with the blue edge. May be kept in place for several days depending on the wound's condition. **Note:** Contact your health care professional at any sign of infection (red, warm, swollen or painful skin). Do not use in combination with ointments or creams. Do not use if the inner packaging is damaged or open. Too frequent dressing replacement may cause irritation to sensitive skin. **Material:** Polyurethane film, polyacrylate, polyester, polypropylene, viscose, Sorbact® fabric based on cellulose acetate.

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ABIGO Medical AB, Ekonomivägen 5
DK ABIGO Pharma A/S, www.abigo.dk
FI Oy Verman Ab, www.sorbact.fi