
## GB SKINOCARE® M3-Nitcomb Lice Comb

Instructions for SkinOcare M3-Nitcomb lice comb with three rows of teeth

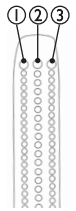The SkinOcare M3-Nitcomb lice comb was developed by pharmacists and has three rows of fine and extra-fine, closely spaced teeth of stainless steel. This patented lice comb with three rows of teeth increases efficiency compared to combs with just one row of teeth. The outer rows of closely spaced teeth remove large head lice. The central row has teeth spaced even closer together, which facilitates the removal of small head lice, eggs and egg shells. The combination of narrow and wide spaces between the teeth doubles the efficiency. The teeth have rounded ends to avoid pulling the hair. The comb is designed with a special angle and a solid handle for easy combing.

### Combing with SkinOcare M3-Nitcomb lice comb

- Water facilitates combing and this method also counteracts tangled hair. Use the SkinOcare M3-Nitcomb lice comb in wet, conditioned hair to remove living and dead lice, eggs and egg shells. Shampoo and rinse. Massage plenty of conditioner into the hair and leave in place.
- Take a small amount of hair. Comb it gently and thoroughly from the root towards the ends. Repeat and comb the entire head of hair in sections, particularly behind the ears and along the back of the neck. You might be able to see fast-running grown lice, but this can be difficult, as they may have the same colour as the hair. It is easier to spot the small, white eggs and shells that are attached to the roots close to the warm scalp, where the female lice deposit their eggs.
- Rinse and clean the comb between the teeth during and after combing with hot, clean soap water using a nailbrush. Finish by rinsing and drying the comb carefully. Repeat combing every 3 to 4 days until the lice are gone.
- Examine the hair regularly. Head lice can easily transfer from one head to another amongst family members and friends in close contact, particularly when the scalp is clean and warm.

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M3-Nitcomb: Patent, ® Design © 2012, Property of Alpa and Hiten Pabari.
Shantys Ltd. RM8 1HJ, UK