# Mepore<sup>®</sup> Pro

STERILE EO

Single use  
Usage unique

CE 0086

MÖLNLYCKE  
HEALTH CARE




USA 1-800-882-4582  
Canada 1-800-494-5134

Manufacturer

Mölnlycke Health Care AB

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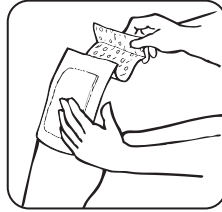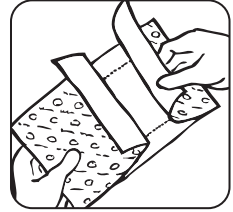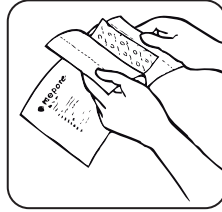Absorbent showerproof adhesive dressing

## Description

The Mepore Pro wound pad is an absorbent viscose pad with a low-adherent wound contact layer. The vapour permeable film provides a showerproof barrier which protects from fluid strike through and bacterial contamination. Mepore Pro conforms well to body contours. The waterbased acrylic adhesive is skin friendly and free from organic solvents. The adhesive area is covered with two overlapping release films enabling aseptic application.

## Indications

Mepore Pro is intended for use on low to moderately exuding wounds, such as surgical wounds, cuts and abrasions.

## Instructions for use

See pictures:

1. Open the sterile pack and remove the dressing.
2. Remove the protection film, exposing the adhesive surface. Position the dressing over the wound without stretching.
3. Firmly smooth the adhesive border onto the skin to obtain proper adhesion.
4. Do not stretch the dressing when applying!

## Frequency of change

The frequency of dressing changes should be governed by the condition of the wound in accordance with accepted clinical practice.

## Precautions

Mepore Pro should not be applied on patients who are sensitive to acrylic adhesive.

Care should be taken to minimise skin tension that could cause mechanical damage to the skin:

1. Ensure that Mepore Pro is not applied under tension.
2. When applying over joints be careful to apply in such a way that it does not impair flexibility of joints.
3. After some surgical interventions excessive swelling may occur. In such a case it could be better to use a more flexible dressing, such as Mepore Film polyurethane film, in order to minimise skin tension.

The wound should be inspected for signs of infection according to clinical practice.

In case of clinical signs of infection (pain, redness, swelling or unusual odour or discharge) continued use of Mepore Pro should be determined by a health care professional.

## Presentation

Each dressing is presented sterile, individually wrapped in a peel pouch.

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**CANADA** Mölnlycke Health Care Inc.,  
2010 Winston Park Drive, Suite 100,  
Oakville, Ontario, L6H 5R7  
Tel: +1 905 829 1502

**CHINA** Mölnlycke Health Care  
(Shanghai) Co., Ltd.  
RM 629, No.8 Hua Jing Road,  
Wai Gao Qiao FTZ Shanghai, China  
Tel: +86 10 5128 8571

**CZECH REPUBLIC** Mölnlycke Health Care s.r.o.  
Hajkova 2747/22, 130 00 Prague 3  
Tel: +420 221 890 517 (**Reception**)  
Tel: +420 221 890 511 (**Customer Service**)

**DENMARK** Mölnlycke Health Care ApS,  
Gydevang 33, DK-3450 Allerød  
Tel: +45 48 16 82 68  
**Customer Service** Tel: +80 886 810

**ESTONIA** Mölnlycke Health Care OÜ,  
Vabaõhumuuseumitee 4-67, EE-13522 Tallin  
Tel: +372 671 1520

**FINLAND** Mölnlycke Health Care Oy,  
Pitäjänmäentie 14, 3 krs, FI-00380 Helsinki  
Tel: +358 201 622 300  
**Customer Service** Tel: 0800 113 442

**FRANCE** Mölnlycke Health Care SAS,  
13 Allée du Château Blanc  
59290 WASQUEHAL  
Tel: +33 320 122 555  
**Customer Service** Tel: 0800 910 292

**GERMANY** Mölnlycke Health Care GmbH  
Grafenberger Allee 297  
DE-40237 Düsseldorf  
Tel: +49 211 92 08 80  
**Customer Service** Tel: 0800 186 21 80

**HUNGARY** Mölnlycke Health Care Kft  
HU-1134 Budapest, Dévai u. 26-28. III. em.  
Tel: +36 1 477 3080

**INDIA** Mölnlycke Health Care India Pvt Ltd  
Universal Business Centre  
Second Floor, Universal Trade Tower  
Sector 49, Gurgaon Sohna Road  
Gurgaon 122 018, Haryana  
Tel: +91 124 4696 008

**ITALY** Mölnlycke Health Care s.r.l.,  
Via Marsala 40/C, IT-21013 Gallarate (VA)  
Tel: +39 0331 714 011

**JAPAN** Mölnlycke Health Care KK,  
4F Concieria Nishi-Shinjuku Tower's West  
6-20-7 Nishi-Shinjuku,  
Shinjuku-Ku, Tokyo 160-0023  
Tel: +81 3 6914 5004

**KOREA** Mölnlycke Health Care Korea Co., Ltd  
2nd floor, Hansol Richville.  
46 Wiryeseong-daero,  
Songpa-gu, Seoul, 05627  
South Korea  
Tel: +82 2 3789 1402

**LATVIA** Mölnlycke Health Care AB,  
Dzelzavas 120 M, Riga LV-1021  
Tel: +372 6711 522

**LITHUANIA** Mölnlycke Health Care AB,  
J. Galvydzio g. 3, LT 08236 Vilnius  
Tel: +370 5 2274 58 37

**NETHERLANDS** Mölnlycke Health Care B.V.  
Postbus 3196, 4800 DD Breda  
Tel: +31 76 521 9663

**NORWAY** Mölnlycke Health Care AS  
Postboks 6229 Etterstad, NO-0603 Oslo  
Tel: +47 22 70 63 70  
**Customer Service** Tel: 800 161 78

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Mölnlycke Health Care AG
Mölnlycke Health Care Ltd.
**USA** Mölnlycke Health Care US, LLC