**caya® diaphragm gel**

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EAN 4013273001250

**caya® diaphragm gel**

**Instructions for use**  
(Please read carefully and keep for future reference.)

**Caya®** is a contraceptive gel when used together with a diaphragm (e.g. Caya® contoured diaphragm) or a cervical cap. Caya® Gel IS NOT a sole contraceptive method when used alone without a mechanical barrier. Caya® Gel is an application method when used alone without a mechanical barrier, **NENI** being compatible with condoms made of natural rubber latex or polyurethane.

**Application:** Before inserting into the vagina, place about one teaspoonful (about 4 ml) of Caya® Gel in the diaphragm or cervical cap. Then follow the instructions for inserting the diaphragm or cervical cap.

The cellulose gel creates an additional mechanical barrier between the diaphragm (or a cervical cap) and the uterine orifice. The lactic acid with its physiological pH of 3.8 has a motility inhibiting effect on the sperms – they are slowed in their forward movement or stopped.

**Composition:** Water, lactic acid, sodium lactate, cellulose, sorbic acid, vanillin.  
**Storage:** Store in a dry place between 4 °C and 25 °C. Always keep the tube closed. For minimum shelf-life, see printing on tube.

**Use within 3 months**  
  
Side effects: Caya® Gel, because of its pH, was well tolerated by the vaginal flora. It can be used for as long and as often as required. If you notice any adverse effects during use, stop using Caya® Gel and ask your doctor before further use.

**Contraindications Caya® Gel**  
There have been no reports of women experiencing side effects during the use of Caya® Gel.

Caya® Gel should not be used if you are known to be hypersensitive or have an allergy to one of the ingredients.

If you experience severe burning or itching during use, stop using the gel and contact your doctor.

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KESSEL medintim GmbH