Used at hospitals as  
Sorbact® Absorption Dressing

### EN Sorbact® Absorption

#### Absorption dressing for exuding superficial wounds

Sorbact® binds bacteria from the wound, which facilitates the healing process and prevents infections. **Usage:** Exuding wounds, such as surgical wounds, acute wounds, pressure ulcers, and foot and leg ulcers. **Instructions:** Clean the wound. Apply the dressing with the green surface in direct contact with the wound surface. The dressing should overlap the wound margins by at least 2 cm. Secure in place with e.g. adhesive tape or a bandage. Do not cut the dressing. May be kept in place for several days depending on the condition of the wound and the surrounding skin. **Note:** Contact your health care professional at any sign of infection (red, warm, swollen or painful skin). Do not use in combination with ointments or creams. Do not use if the inner packaging is damaged or open. If the dressing dries out or adheres to the wound, it can be moistened or soaked in water or 0.9% saline solution to facilitate removal and to avoid interference with the healing process. **Material:** Viscose, cellulose, polyester, cotton, Sorbact® fabric based on cellulose acetate.

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ABIGO Medical AB, Ekonomivägen 5
DK ABIGO Pharma A/S, www.abigo.dk
FI Oy Verman Ab, www.sorbact.fi