

DASH 3™**DASH 3™
Plus****INTERVENE**

DASH 3™ - ENFit™ Syringe

INSTRUCTION FOR USE

INTENDED USE


DASH 3™ - ENFit™ Syringe is intended for delivery of liquid medication and formula enteral feeding nutrition, aspiration and flushing.

INTENDED USER

DASH 3™ - Single Use ENFit™ Syringe is intended to be used by Medical Personnel within hospitals and for individuals at home.

DASH 3™ Plus - Reusable ENFit™ Syringe is intended for Single Patient Use and home care only.

HOW TO USE ENTERAL LOW DOSE SYRINGES TO AVOID OVERDOSING

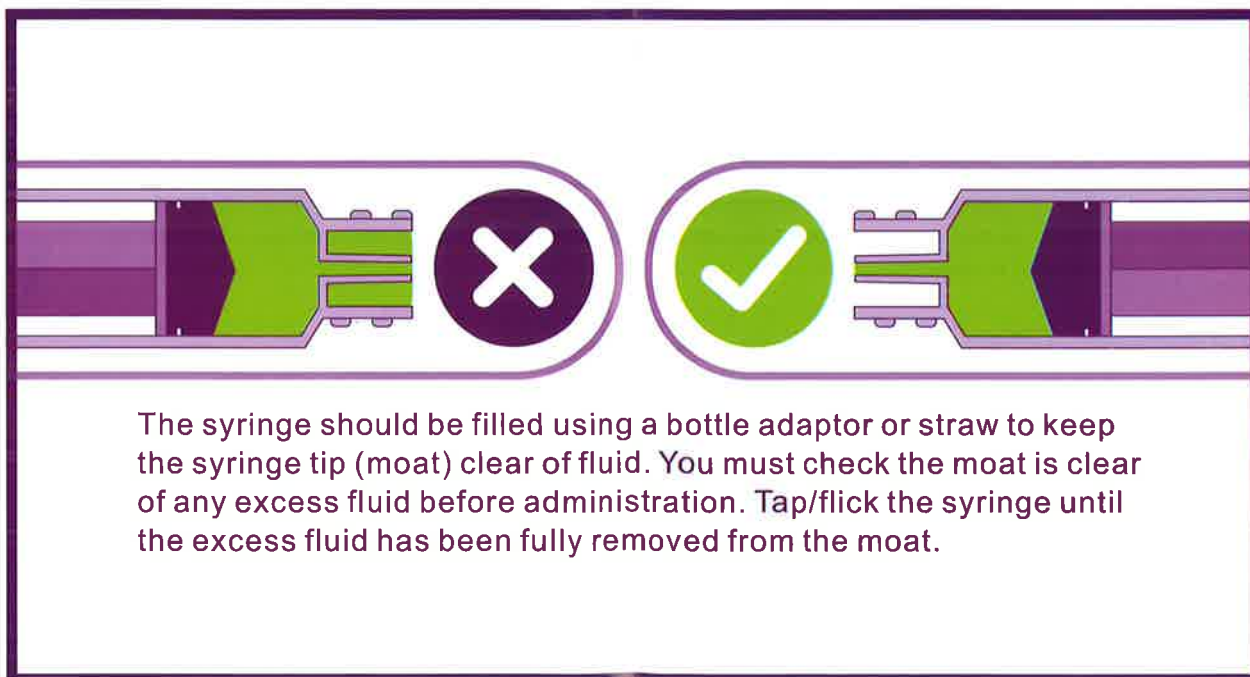

INTERVENE, RUSSELL BUILDING, BRUNEL SCIENCE PARK, KINGSTON LANE, UXBRIDGE
UB8 3PQ, UNITED KINGDOM

dash3.co.uk

CE
0120

DASH 3™**DASH 3™**
*Plus***INTERVENE**

DASH 3™ - ENFit™ Syringe

INSTRUCTION FOR USE

WASHING THE DASH 3™ Plus - Reusable ENFit™ Syringe

- Step 1:** Wash syringes immediately after use. **DO NOT** allow contents to dry in a used syringe as this will shorten the effective life of the product.
- Step 2:** Withdraw the plunger fully from the syringe barrel. It is not necessary to remove the black rubber gasket from the plunger rod.
- Step 3:** Rinse both parts (the barrel and plunger assembly) in warm soapy water (household detergent) to remove any particulates from the last feed.
- Step 4:** **DO NOT** scrub any of the syringe components.
- Step 5:** Rinse off any soapy water under clean running water or in a bowl of clean water.
- Step 6:** Allow the syringe parts to dry before reassembly.
- Step 7:** **DO NOT** towel dry or rub dry the syringe parts as this will shorten the effective life of the product.
- Step 8:** Store the syringe in a clean place away from direct sunlight and moisture between uses.
- Step 9:** The syringe may be used for up to seven consecutive days; until the graduations become unclear on the barrel or until the plunger becomes too stiff to operate; whichever is the shorter period of time.

DISPOSAL OF DASH 3™ - ENFit™ Syringe

Hospital Use: Must be disposed of as per hospital protocols

Home Use: To be disposed in household rubbish/trash

Symbology Glossary



Manufacturer



Expiry Date



Sterilized using ethylene oxide



Caution




Catalogue Number



Consult Instruction for Use



Keep Dry

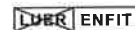

Non-Luer ENFit connection system



Batch number



Do not use if package is damaged



Keep away from sunlight



Date of Manufacture



Do not re-use



Non-pyrogenic



INTERVENE, RUSSELL BUILDING, BRUNEL SCIENCE PARK, KINGSTON LANE, UXBRIDGE
UB8 3PQ, UNITED KINGDOM

dash3.co.uk

CE
0120