EasySampler®

Complete Stool Collection Kit

 Brugsanvisning

 Bruksanvisning

 Instructions for use

www.EasySampler.com

Trin 1: Forberedelse og montering af EasySampler

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Step 1: Preparation and mounting of the EasySampler

Put up the toilet seat before EasySampler is mounted.

Dry off the toilet bowl so the EasySampler can be mounted on a clean and dry surface.

Unfold the EasySampler.


EasySampler may not touch the surface of the water.

Remove the two tape protections from the EasySampler and mount them on the back of the bowl.


Remove the remaining tape protection and mount the front part of the EasySampler at each side in mid position of the bowl, leaving space to dispose toilet paper in front of EasySampler.

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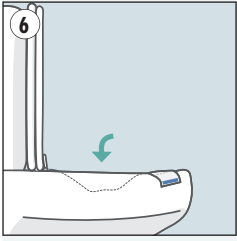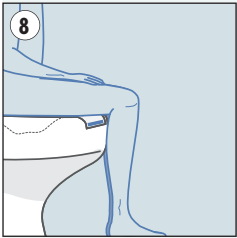**Important:**

EasySampler is mounted so it creates a sample area.

Finally put down the toilet seat. EasySampler is now ready for use.

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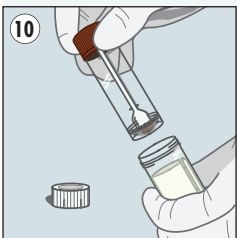
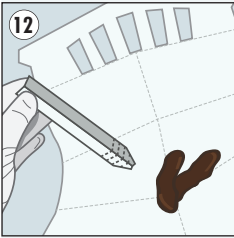
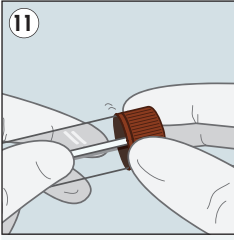
**Step 2:
Collecting
the stool sample**

Put on the Gloves.


Open the outer container and take out the sample tube. Leave the absorbent material in the outer container.

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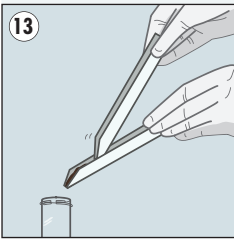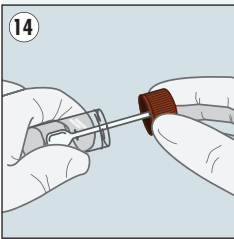
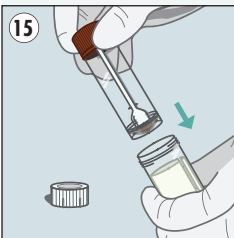
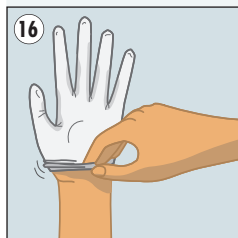
Open the sample tube.

Take up the faeces sample with one of the paperspoons.


Push down the faeces sample in the sample tube. Repeat this till you have the amount of faeces your Doctor requested.

Close the sample tube so it will not leak during transportation. Put the sample tube into the outer container.


Place sample container into the outer container keeping the absorbent material between the two containers. Screw the lid onto the outer container.

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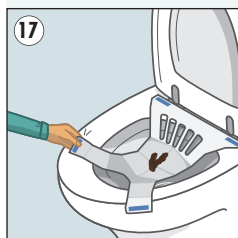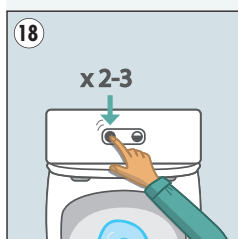Twist the gloves over the paperspoons and leave it all with other household garbage."

Loosen EasySampler from the Bowl and flush it out in the toilet.


When using water-saving toilets use full flush. Repeat it until the amount of water are 10-15 liter. Tape leftovers on the bowl can easily be removed with denatured spirit.

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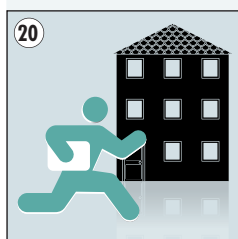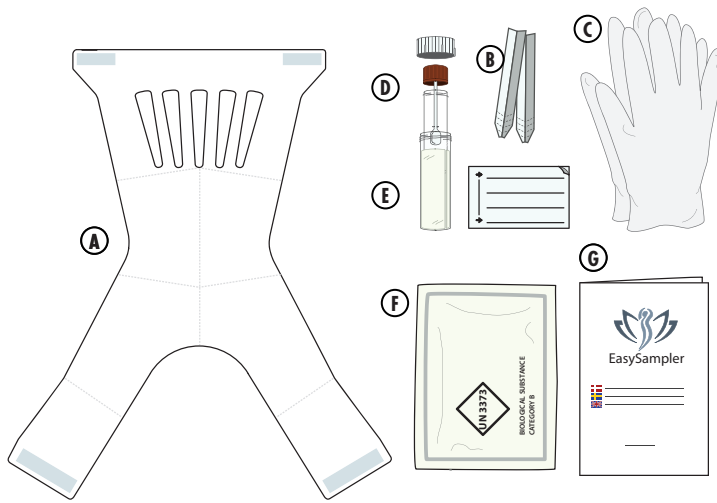
Step 3:

Packing and sending the stool sample

Place the sealed container into the padded envelope.

Mail or carry the sample package to laboratory as soon as possible, as per your instructions.

EasySampler is available in the following packages:

Item no.	Description	
10 01 00	EasySampler – Single 1 x EasySampler 1 x Instructions for use	A G
10 60 10	EasySampler – Mini Kit - Single 1 x EasySampler 2 x EasySpoon 2 x Gloves – Nitrile – Size Large 1 x Instructions for use	A B C G
10 70 00	EasySampler – Complete Stool Collection Kit 1 x EasySampler 2 x EasySpoon 2 x Gloves – Nitrile – Size Large 1 x Sample tube 1 x Shipment tube and label 1 x Envelope <UN3373> 1 x Instructions for use	A B C D E F G

EasySampler are distributed by