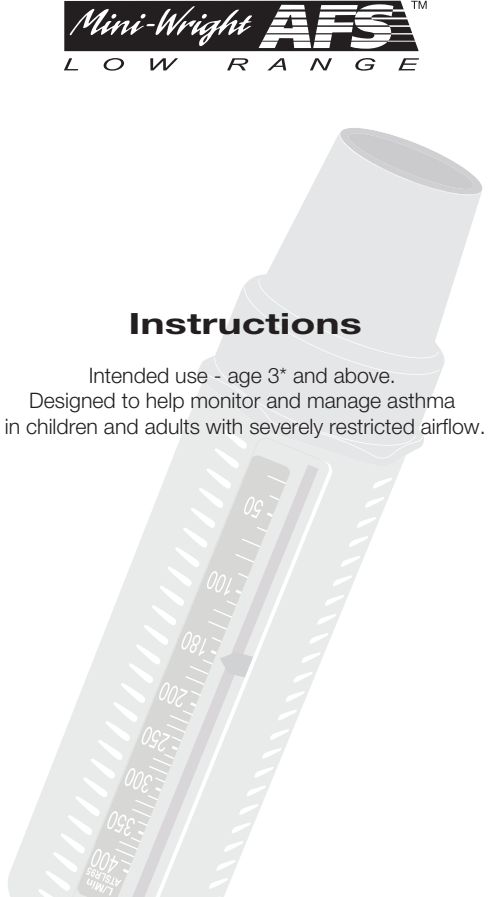
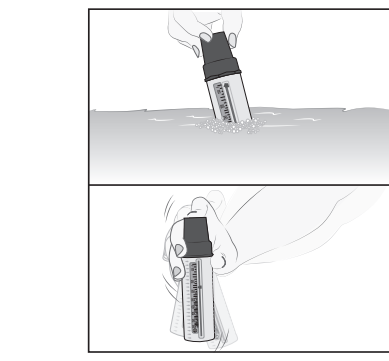
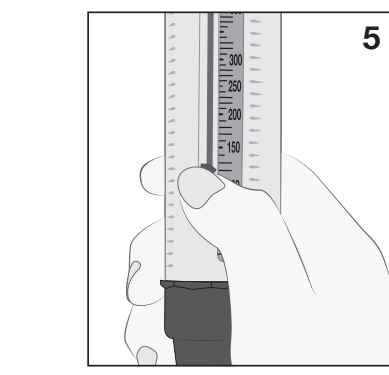
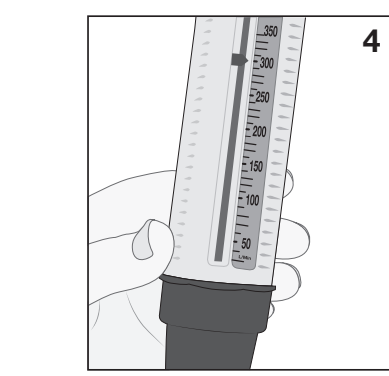
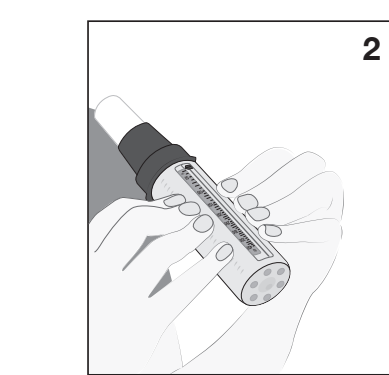
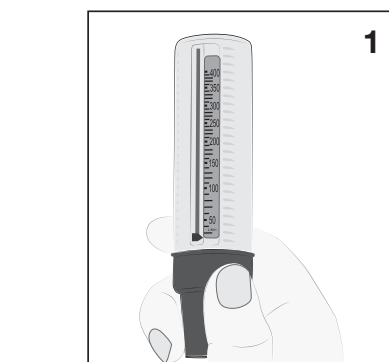
Low Range Peak Flow Meter conforms to BS EN ISO 23747 for range 30 to 400 L/min (EU Scale)
Universal Range 30 - 400 LPM

* windmill trainer designed to encourage children

en

Your Mini-Wright Low Range Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use and care for your peak flow meter and how to record the readings. Peak flow measurements are an indication of how well air is moving through your lungs. Changes in your peak flow may require special treatment to establish what is happening in your airways.

With careful use the Mini-Wright low range peak flow meter would be expected to last for two years (2,000 uses).

Contents:

- MINI-WRIGHT Low Range peak Flow Meter, complete with detachable universal mouthpiece and instructions with peak flow diary.
- Do not obstruct the holes at the end of the peak flow meter.
- Please note that any coughing or spitting into the meter will adversely affect your readings and should be avoided.
- Fungal growths may occur if the instrument has not been cleaned for some time.
- Store in a clean dry place.

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Disposal Note:

The Mini-Wright peak flow meter can be disposed of as normal household waste.

How to Use Your Mini-Wright Low Range Peak Flow Meter

- Insert the mouthpiece into the mouth. Ensure the pointer is at the bottom of the scale (nearest the mouthpiece).
- Hold the peak flow meter so that your fingers are clear of the scale and slot.
- Stand up if possible. Take a deep breath, place the peak flow meter in the mouth and hold horizontally, closing the lips around the mouthpiece, then blow as hard and as fast as you can.
- Note the number on the scale indicated by the pointer on a piece of paper. If unusual readings are obtained and the meter does not appear broken – seek advice from your healthcare professional.
- Return the pointer to the bottom of the scale (nearest the mouthpiece) and repeat the procedure twice more to obtain three readings. Mark the highest of the three readings on your peak flow diary.

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Cleaning Instructions*

- Immerse the peak flow meter in warm (but not hot) mild detergent solution for 2-3 minutes (maximum 5 minutes).
- Agitate the meter to ensure thorough cleaning.
- Rinse in clean warm water and shake gently to remove any excess water.
- Allow it to dry thoroughly before using again.
- The mouthpiece is detachable and should be kept clean using a proprietary disinfectant solution.

Performance Accuracy

The Mini-Wright low range meter conforms to BS EN ISO 23747 (for the range 30 to 400 L/min) and the NAEPF guidelines.

Accuracy: Better than 10% or 10 L/min. Readings expressed as BTPS values. Resistance to flow does not exceed 0.36 kPa/l/s.

* For Multi-Patient use see Care and Use

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