# MENSTRUATIONSKOP

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## ENGLISH (EN)

Vivag Menstruation Cup is made of 100% silicone for medical use and is clinically tested, so it does not disturb the natural balance of the body.

### DESCRIPTION OF VIVAG MENSTRUAL CUP

- A) Rim
- B) Perforations to break the suction effect during removal
- C) External ridges
- D) Reinforced base
- E) Stem

### WHAT SIZE SHOULD I CHOOSE?

Size 1- Light to normal flow:  
L: 74 mm; Ø: 41 mm; Capacity: 20.2 ml

Size 2 - Heavy flow:  
L: 74 mm; Ø: 46 mm; Capacity: 29.5 ml

## WE RECOMMEND THAT YOU READ THESE INSTRUCTIONS BEFORE USE

### GENERAL WARNINGS AND PRECAUTIONS

- Do not use VIVAG MENSTRUAL CUP if you are allergic to any of its components.
- If you experience itching or burning, urinary discomfort or pain, stop using the product immediately and contact your doctor.
- Do not use VIVAG MENSTRUAL CUP if you have a vaginal infection.
- If you use an IUD or contraceptive ring, ask your doctor prior to using VIVAG MENSTRUAL CUP and break the vacuum seal.
- Do not use VIVAG MENSTRUAL CUP in association with other intravaginal devices or treatments (tampon, lubricant, capsule...).
- After childbirth, surgery, miscarriage or abortion, or if you have a vaginal wound, give your vagina time to heal before using VIVAG MENSTRUAL CUP.
- VIVAG MENSTRUAL CUP does not protect against sexually transmitted diseases (STDs).
- VIVAG MENSTRUAL CUP is only to be worn during your period.
- VIVAG MENSTRUAL CUP is not a contraceptive.
- Remove VIVAG MENSTRUAL CUP prior to sexual intercourse.
- VIVAG MENSTRUAL CUP is intended for personal use only.
- Remove VIVAG MENSTRUAL CUP before any medical exam, including medical imaging exams.
- Keep VIVAG MENSTRUAL CUP out of reach of children.
- Keep VIVAG MENSTRUAL CUP out of reach of animals.
- The hymen may not be preserved with the use of VIVAG MENSTRUAL CUP.
- If you have long nails, be careful not to wound yourself when inserting and removing your VIVAG MENSTRUAL CUP.

### WHY IS VIVAG MENSTRUAL CUP THE CUP FOR ME?

VIVAG MENSTRUAL CUP is a menstrual cup made of biocompatible medical grade silicone. VIVAG MENSTRUAL CUP comes in its case, which is used to carry, clean and disinfect VIVAG MENSTRUAL CUP. VIVAG MENSTRUAL CUP does not alter vaginal secretions, so you can safely wear it throughout your period, whether your flow is light or heavy. VIVAG MENSTRUAL CUP can be reused, day and night, during travel and physical activities. VIVAG MENSTRUAL CUP is your cup. It is a personal, intimate care product that you can store in the case when you are not on your period. **Warning: VIVAG MENSTRUAL CUP should only be worn during menstruation.**

### WHAT ARE THE PRECAUTIONS FOR USE?

- Wash your hands thoroughly before use.
- It is important to always have clean hands with short nails before handling the VIVAG MENSTRUAL CUP.
- Open the bag, take out the VIVAG MENSTRUAL CUP then discard the bag.
- Check the integrity of the VIVAG MENSTRUAL CUP before each use.
- Clean your VIVAG MENSTRUAL CUP with cold water and Vivag Perfume Free Intimate Wash® and rinse thoroughly with water.
- Disinfect your VIVAG MENSTRUAL CUP:
  - Option 1: Place your VIVAG MENSTRUAL CUP stem down in the case, fill with water up to the mark FIGURE 1. Leave the lid open and heat the case in a 750 Watts microwave oven for 5 minutes.
  - Option 2: plunge your VIVAG MENSTRUAL CUP into a pot of boiling water for 7 minutes.

Your VIVAG MENSTRUAL CUP is now disinfected. Wait for your VIVAG MENSTRUAL CUP to cool before use to avoid the risk of burns or damaging the silicone.

### HOW DO I INSERT MY VIVAG MENSTRUAL CUP?

Wash your hands before handling your VIVAG MENSTRUAL CUP. Warning, do not use lubricant to insert your cup. • Fold your VIVAG MENSTRUAL CUP into a punch down fold or a U shape as shown in FIGURES 2.1 and 2.2 and hold it firmly between your thumb and index finger as close to the base as possible while keeping it folded. • With your other hand, gently separate your labia and insert your VIVAG MENSTRUAL CUP approximately one centimetre from the vaginal entrance. • Release your grasp and your VIVAG MENSTRUAL CUP will automatically position itself at the base of the vagina FIGURE 3, unlike tampons which are positioned at the top of the vagina.

### HOW DO I REMOVE MY VIVAG MENSTRUAL CUP?

- Wash your hands before handling your VIVAG MENSTRUAL CUP.
- Your VIVAG MENSTRUAL CUP must be emptied every 6 hours, or every 2 to 3 hours for heavier flow.
- Sit down comfortably on the toilet to remove your VIVAG MENSTRUAL CUP (your VIVAG MENSTRUAL CUP is held in place by your vaginal muscles. If you are tense, it will be harder to remove the VIVAG MENSTRUAL CUP).
- You must first release the seal: use the stem of the VIVAG MENSTRUAL CUP as a guide to find the base. Pinch the base between two fingers or insert a finger between the vaginal wall and the VIVAG MENSTRUAL CUP to release the seal FIGURES 5.1 and 5.2.
- You can now remove your VIVAG MENSTRUAL CUP by gently pulling on the base FIGURE 4.
- Empty the contents into the toilet.
- Clean your VIVAG MENSTRUAL CUP before reinsertion.

### WHAT TO DO IF YOU EXPERIENCE DISCOMFORT OR LEAKS?

- If you experience any discomfort while walking or while seated, or feel a pinching sensation in your vagina, you should trim your VIVAG MENSTRUAL CUP stem to the correct length for your vagina. Take time to ensure a proper fit. Trim the stem incrementally with clean scissors and reinsert your VIVAG MENSTRUAL CUP to ensure that the stem fits comfortably inside your vagina FIGURES 6.1, 6.2 and 6.3.
- In case of leakage, reposition your VIVAG MENSTRUAL CUP one centimetre from the vaginal entrance. Check the length of the stem.
- Check the vacuum seal on your VIVAG MENSTRUAL CUP inside your vagina in two simple steps FIGURES 7.1, 7.2 and 7.3:
  - Step 1: Slide one finger around your VIVAG MENSTRUAL CUP to ensure that it opened properly inside your vagina.
  - Step 2: Pinch the base of your VIVAG MENSTRUAL CUP to rotate it inside your vagina.

### HOW DO I CLEAN AND CARE FOR MY VIVAG MENSTRUAL CUP?

- It is important to always keep your VIVAG MENSTRUAL CUP clean.
- Clean your VIVAG MENSTRUAL CUP before each insertion and at the end of menstruation with Vivag Perfume Free Intimate Wash®, Never use petroleum products (Vaseline), essential oils, corrosive cleaners, perfumed soaps or dish soap to clean your VIVAG MENSTRUAL CUP.
- Rinse your VIVAG MENSTRUAL CUP thoroughly with clean water.
- Desinfect your VIVAG MENSTRUAL CUP before each insertion and at the end of each cycle.

You have two options:

- Option 1: Place your VIVAG MENSTRUAL CUP stem down in the case. Fill with water up to the mark. Leave the lid open and heat the case in a 750 Watts microwave oven for 5 minutes FIGURE 1.
- Option 2: Plunge your Vivag Menstruationsskop into a pot of boiling water for 7 minutes.

• At the end of your menstrual cycle, clean and disinfect your VIVAG MENSTRUAL CUP. Then dry your VIVAG MENSTRUAL CUP in open air to prevent odours. Store your VIVAG MENSTRUAL CUP in its case until your next period. If you notice any damage or change in the appearance, smell or texture of your VIVAG MENSTRUAL CUP, you should replace it. Under the recommended conditions for use and maintenance, your VIVAG MENSTRUAL CUP can be used for 3 years.

**COMPOSITION:** Platinium bio-compatible medical-grade silicone.

### INFORMATION ABOUT TOXIC SHOCK SYNDROME (TSS)

TSS is a rare, but potentially serious, or even fatal infection. It is caused by a toxin produced by bacteria (Staphylococcus) commonly found in the body. Symptoms of TSS may appear suddenly during your period or immediately after. Symptoms are similar to those associated with the common flu and include: sudden high fever (>39°C), muscle aches, vomiting, diarrhoea, headache, dizziness, fainting, or skin rashes similar to sunburn. If you experience any of these symptoms while using VIVAG MENSTRUAL CUP remove it and contact your doctor immediately. Do not use VIVAG MENSTRUAL CUP or any other intravaginal device if you have suffered from TSS or if you are severely immunocompromised.

For more information, please contact us via email at: [vivag@orkla.dk](mailto:vivag@orkla.dk). Please include the lot number printed on your VIVAG MENSTRUAL CUP packaging.

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