INSTRUCTIONS FOR USE:

Indications for use: PEDIACYN™ is indicated as an adjuvant in the wound healing process with wounds that can only heal by secondary intention in maturation phase. PEDIACYN™ is indicated for the care of lesions associated with atopic dermatitis. PEDIACYN™ assists with moistening while forming a protective barrier against physical, chemical and microbial invasion of the atopic dermatitis lesions.

Handling: PEDIACYN™ is biocompatible for its indicated use, non-irritating to human skin and is not a skin sensitizer. No special handling precautions are required.

Storage: PEDIACYN™ should be stored in its original sealed container at ambient conditions. Store at room temperature and prevent product from freezing. It is non-flammable. After use, the container should be closed again, before storing.

Stability: Until opened, the stability of PEDIACYN™ is assured up to expiration date (see bottle for year and month of expiration YYYY MM). Upon opening, note the date on the bottle in the section "Date opened". Stability of the unused hydrogel is assured for up to 60 days after opening.

Directions for use: PEDIACYN™ is ready for use with no mixing or dilution required. Prior to cleanse the area or use the product, please carried out an appropriate handwashing. Apply three times daily.

Treatment: Prior to cleanse the area or use the product, please carried out an appropriate handwashing. Wash the affected area as recommended by your healthcare professional. Dry. Cover the affected area with a thin layer of PEDIACYN™ and gently rub until it is evenly distributed. Let skin dry and do not wash off.

Warnings and points to note: • If tamper resistant seal is broken, do not use. • For external use only, not for injection. Avoid direct contact with eyes, mucous membranes, third degree burns and open wounds. • Do not use if sensitive to polydimethylsiloxane. • Do not use PEDIACYN™ in combination with other topical products without the advice of a healthcare professional. • In case of deep or serious wounds or if you have any other treatment concerns consult a healthcare professional. • If signs of infection consult a healthcare professional. • Report any adverse reactions to a healthcare professional. • Keep out of the reach and out of a sight of children.
--

Disposal: No special disposal considerations required. Follow local regulation for unused and expired pharmaceutical products.

Ingredients: Electrolysed Water (H₂O), Poly-dimethylsiloxane, Sodium Magnesium Fluorosilicate, Sodium Phosphate, ancillary substances: Sodium Hypochlorite (NaOCl) and Hypochlorous Acid (HOCl).

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