Instructions for use:

EN

Indications for use: Microdacyn[®]₆₀ Oral Care is an electrolysed solution intended for use as an adjuvant for the treatment of mouth and throat infections and the debridement and moistening of mouth lesions and thrush. Microdacyn[®]₆₀ Oral Care can be broadly applied within a comprehensive oral cavity treatment.

Handling: Microdacyn[®]₆₀ Oral Care is biocompatible for its indicated use, non irritating to human skin and is not a skin sensitizer. No special handling precautions are required.

Storage: Microdacyn[®]₆₀ Oral Care should be stored in its original sealed container at ambient conditions. Store at room temperature and prevent product from freezing. It is non flammable. After use, the container should be closed again, before storing.

Stability: Until opened, the stability of Microdacyn[®]₆₀ Oral Care is assured up to expiration date (see bottle for year and month of expiration “YYYY MM”). Upon opening, note the date on the bottle in the section “Date opened”. Stability of the unused solution is assured for up to 60 days after opening.

Directions for use: Microdacyn[®]₆₀ Oral Care is ready for use with no mixing or dilution required. Apply three times a day for as long as healthcare professional care tells you or until there are no inconvenience.

Treatment: You can use it per the instructions provided by your dentist.

Warnings and points to note:

- If tamper resistant seal is broken or missing, do not use.
- Do not use if hypersensitive to product ingredients
- Do not use Microdacyn[®]₆₀ Oral Care in combination with other oral care products without the advice of a healthcare professional.
- In case of deep or serious wounds or if you have any other treatment concerns consult a healthcare professional.
- If signs of infection consult a healthcare professional.
- Report any adverse reactions to a healthcare professional.
- For use in children less than 7 years must be supervise by an adult.
- Keep container closed when not using.
- Microdacyn[®]₆₀ Oral Care is harmless even in case of accidental contact with eyes, nose or ingestion.

Disposal: Microdacyn[®]₆₀ Oral Care is non-toxic and safe to the environment. No special disposal considerations required.

Ingredients: Electrolysed Water (H₂O), Sodium Chloride (NaCl) ancillary substances: Sodium Hypochlorite (NaOCl) and Hypochlorous Acid (HOCl).

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