Your Mini-Wright Low Range Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This booklet explains how to use and care for your peak flow meter and how to record the readings. Peak flow measurements are an indication of how well air is moving through your lungs. Changes in your peak flow may require special treatment to establish what is happening in your airways. With careful use the Mini-Wright low range peak flow meter would be expected to last two years (2,000 uses).

#### Contents:

1 MINI-WRIGHT Low Range Peak Flow Meter complete with detachable universal mouthpiece and instructions with peak flow diary.

- Do not obstruct the holes at the end of the peak flow meter.
- Please note that any coughing or spitting into the meter will adversely affect your readings and should be avoided.
- Fungal growths may occur if the instrument has not been cleaned for some time. Store in a clean dry place.

## Instructions

Intended use - age 3\* and above.
Designed to help monitor and manage asthma in children and adults with severely restricted airflow.

<b>en</b>	<b>العنوان</b>
<b>ar</b>	<b>الموزع</b>
<b>ca</b>	<b>توزيع</b>
<b>de</b>	<b>Hersteller</b>
<b>es</b>	<b>Compañía</b>
<b>fr</b>	<b>Le fabricant</b>
<b>hr</b>	<b>Proizvođač</b>
<b>it</b>	<b>Fabbricante</b>
<b>nl</b>	<b>Fabrikant</b>
<b>pl</b>	<b>Producent</b>
<b>pt</b>	<b>Fabricante</b>
<b>ru</b>	<b>Производитель</b>
<b>sv</b>	<b>Fabrikant</b>
<b>tr</b>	<b>Üretici</b>
<b>uk</b>	<b>Виробник</b>
<b>vi</b>	<b>Nhà sản xuất</b>
<b>zh</b>	<b>製造商</b>
<b>zj</b>	<b>Producent</b>

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#### Disposal Note:

The Mini-Wright peak flow meter can be disposed of as normal household waste.

#### How to use Your Mini-Wright Low Range Peak Flow Meter

- Insert the mouthpiece into the meter. Ensure the pointer is at the bottom of the scale (nearest the mouthpiece).
- Hold the peak flow meter so that your fingers are clear of the scale and slot.
- Stand up if possible. Take a deep breath, place the peak flow meter in the mouth and hold horizontally, closing the lips around the mouthpiece, then blow as hard and as fast as you can.
- Note the number on the scale indicated by the pointer on a piece of paper. If unusual readings are obtained and the meter does not appear broken – seek advice from your healthcare professional.
- Return the pointer to the bottom of the scale (nearest the mouthpiece) and repeat the procedure twice more to obtain three readings. Mark the highest of the three readings on your peak flow diary.

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**How to Record Your Peak Flow**

It is important to keep a record of your peak flow. Three readings should be taken twice a day, first thing in the morning and at bedtime unless otherwise directed by your healthcare professional.

**Mark the highest of the three readings on the peak flow diary provided, both in the morning and at bedtime, every day. By joining the dots (see diagram\*\*) you can see how your peak flow varies from day to day.**

Make sure to measure and note your peak flow if you suddenly become wheezy or breathless at any time of the day or night.

#### Care and Use

This peak flow meter does not require maintenance (other than cleaning) or calibration for the two years following its first use. At the end of two years, it should be replaced.

When not in use this device should be kept in a clean, dry area away from any hazardous substances. It is important to keep your peak flow meter clean when in use. It should be cleaned at least once every six months and thoroughly if it has not been used for some time.

The Mini-Wright low range peak flow meter is suitable for multi-patient use if Clement Clarke's validated cleaning instructions are followed.

These are available on request, or via the internet at: www.clement-clarke.com

#### Troubleshooting

Should you take your peak flow measurement and get an unusually low reading, it could mean that your peak flow meter is broken or it could mean that the reading is wrong and your asthma is getting worse. Check to make sure that the meter is not broken by sliding the pointer along the slot, making sure that it moves freely.

If you feel comfortable that you are following the instructions correctly and that your meter is not broken, contact your healthcare professional for advice. No matter what your peak flow readings are, if you have symptoms such as chest tightness, shortness of breath, coughing or wheezing, you should contact your healthcare professional for advice.

\* For Multi-Resort use See Care and Use

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