LentisCath™

Instruction for use

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(GBR)

Indications for use - LentisCath™ Ready to use intermittent Urinary Catheter is intended for urological use only. It is intended for use by patients for bladder management including urine drainage, collection, and measurement. The device is passed to the urinary bladder via the urethra.

Product name, model - LentisCath™ Ready to use, Hydrophilic catheter ready to use

Sterility status - This product is sold only sterile

Storage conditions - Keep away from sunlight, store at 10°C-25°C, keep dry

Contraindications - None known

Warnings - This is a single use device. Do not resterilize any portion of this device. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness or death of the patient.

Safety measures - Inspect the catheter before use. Do not use the product if the device or packaging is damaged. Please consult product inserts and labels for any indications, contraindications, hazards, warnings, cautions and directions for use.

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