Company : Owen Mumford Plc Ltd
 Particular : 39160-AD-GL-IFU-P004486A1-01-HR
 F.I. No. : 2412421
 Actual Size : Open : (W) 276 x (H) 160 (mm)
 Close :
 Date : 01102024
 Rev. Date : 00.00.0000

Prepare by
 SIP

Colour

Reference

Artwork Confirmation

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Prepared by Checked by Confirmed By

Signature :
 PIC : Eileen
 Date : 22/03/2022

* Enclose herewith our "ARTWORK" for your perusal and approval. Kindly confirm acceptance by chop & sign & refaxing same to us soonest possible. Thank you.

276mm

46mm

FOLD

46mm

FOLD

46mm

FOLD







46mm

FOLD

46mm

FOLD

46mm

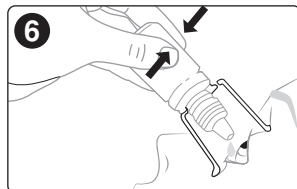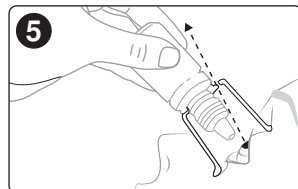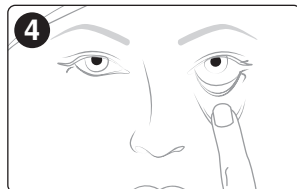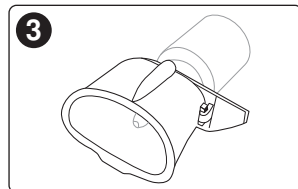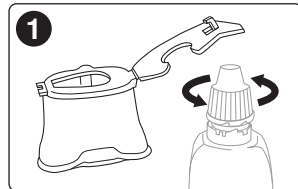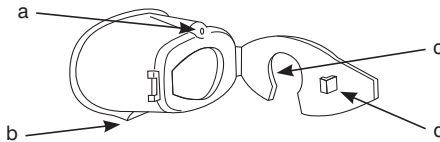ENGLISH:

Instructions for use

AutoDrop® has been developed to assist in the instillation of eye drops. AutoDrop enables patients to self-administer their eyedrops with ease, helping with the delivery of treatments.

AutoDrop is compatible with a number of eye drop devices on the market. For a full up to date list, please visit www.owenmumford.com.

Components:

- a. Pin-hole
- b. Lip
- c. Keyhole slot
- d. Clip

1. Open AutoDrop. Loosen cap from eye drop bottle but do not remove completely.
2. Locate the keyhole slot around the base of the bottle neck below the thread. Remove bottle cap.
3. Close AutoDrop. Do not force the bottle nose into AutoDrop, it should close easily.
4. Hold down lower eyelid. Place AutoDrop over the eye with the lip against the cheek.
5. Tilt the head backwards. Look through pin-hole towards the light.
6. Squeeze bottle gently to allow prescribed drops to fall into the eye.
7. Open AutoDrop and replace top on eye drops bottle between doses.
8. To clean AutoDrop remove eye drop bottle and wash AutoDrop in warm soapy water. Rinse and dry thoroughly.

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46mm

ENGLISH:

We recommend that AutoDrop is used for the whole course of treatment for an eye disorder. When an eye drop bottle is discarded remove AutoDrop and clean thoroughly before fitting to the new bottle. AutoDrop may be used on more than one type of eye drop at the same time but if you suffer from more than one eye complaint we recommend that you use a separate AutoDrop for each type of eye drop.

In hospitals and clinics: AutoDrop may be autoclaved on a low pressure cycle for 30 minutes. Release clip to allow moisture to evaporate. AutoDrop can also be cleaned in sterilising fluid. Rinse and dry thoroughly.

GUARANTEE:

AutoDrop is guaranteed for 3 months from purchase. If AutoDrop breaks within this period a full refund will be given.

If you experience any problems with this medical device please consult a healthcare professional.

If any serious incident occurs with the use of this device please report to Owen Mumford Limited and your local competent authority.

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AutoDrop® is a registered trademark of Owen Mumford Ltd.
© Copyright Owen Mumford 2024.
Owen Mumford Ltd.,
Owen Mumford GmbH,