Please read instructions for use carefully before using the CANNASEN® CBD Protective Nasal Gel.

How does CANNASEN® CBD Protective Nasal Gel work?

CANNASEN® CBD Protective Nasal Gel is developed to provide a long-lasting moistening effect and protect the nasal mucous membrane (i.e., mucosa). Upon applying the gel into the nose, a thin film is formed to protect against pollen and irritation. The product physically works against typical symptoms such as rhinitis, hay fever, inflamed nasal mucosa and blocked nose.

CANNASEN® CBD Protective Nasal Gel is a medical device. Effectiveness and safety have been tested in accordance with the European Medical Device Directive (93/42 / EEC).

Active Principle of CANNASEN® CBD Protective Nasal Gel

Mucous membranes are made up of epithelial cells. The epithelial cells secrete mucus and form a protective mucus layer. Furthermore, the cells filter the inhaled air and help protect against external factors such as dust, pollen, environmental pollution, bacteria and viruses. **CANNASEN® CBD Protective Nasal Gel** is a nasal formulation with a triple active complex. In addition to protecting the nasal mucous membrane, it also moisturizes the mucous membrane and, due to its increased viscosity, can maintain a protective film for longer. **CANNASEN® CBD Protective Nasal Gel** is an isotonic, salty, water-based, moisturizing gel for the nose. The product frees dry and irritated nasal passages, which are triggered by allergies, dust, and low humidity as well as blockages.

Application areas

For long-lasting moistening of the nasal mucosa. Protects and moisturizes the mucous membrane. Gel film to protect against pollen and irritation

Dosage instructions, type and duration of application

For adults and adolescents, the gel can be used up to 2 times a day. There should be at least a 6 hour break between applications. Insert the nasal applicator carefully into your nose. It should not be inserted too deeply in order to avoid injuries! If too much of the product has been accidentally used and is perceived as unpleasant, the excess amount can be removed by blowing your nose vigorously into a tissue. Please consult your doctor if the symptoms persist for more than a week.

Please note

Keep out of reach of small children. Not suitable for children under 14 years of age. Use only after consulting your doctor. During pregnancy or lactation, use only after consultation with a doctor. Do not take the nasal gel orally. Avoid eye contact.

Contraindications

CANNASEN® CBD Protective Nasal Gel must not be used in case of known hypersensitivity to any of the ingredients. Please do not use the product on the nasal mucosa at the same time as other products. Temporary burning, stinging, sneezing or increased runny nose may occur with nasal gels. This indicates that the product is working. These signs are harmless and usually do not last long. However, if the signs persist, discontinue using the product. Occasionally, unpleasant reactions such as blocked or runny nose, watery eyes, local pain, dryness, itching, or swelling are known to occur when using nasal gels. If you are concerned that you are allergic, stop using it immediately. Consult a doctor if symptoms persist. A doctor should also be consulted if the cold has not subsided after 7 days. For reasons of hygiene and to avoid the transmission of pathogens, a nasal gel tube should only be used by one and the same person and the applicator should be cleaned after each application with a clean tissue.

Interactions

To avoid drug interactions, you should use medication 2 hours apart or consult your doctor.

Incident reporting

If this product is associated with a serious incident (death of a patient, user or other person; temporary or permanent serious deterioration of the health status of a patient, user or other person; serious risk to public health), this incident must be reported to the manufacturer and the National competent authorities.

Instructions and information on the durability of the medical device

CANNASEN® CBD Protective Nasal Gel may only be used up to the date marked on the carton with "use by".

Dosage form and content

Original packaging with 10 ml gel.

Composition

Active ingredient complex: Hydroxyethyl Cellulose, Sodium Chloride, Hyaluronic Acid. Other ingredients: Water, Propylene Glycol, Pentylene Glycol, Butylene Glycol, Lauryl Glucoside, Panthenol, Allantoin, Cannabidiol(CBD), Phytic Acid, Sodium Hydroxide. Do not use the product if you are allergic to any of the ingredients.

Storage

Keep cool (between 10°C and 25°C), dry and protected from sunlight. After using the gel, keep the tube well closed. Open tubes must be used within 3 months.

Disposal

The product can be disposed of with normal household waste.

 Manufacturer: CanNordic a/s Fruebjergvej 3 DK-2100 Copenhagen	 Symbol CE	 Upper / lower temperature limit	 Use by	 Please observe the instructions for use	 Batch number	 Protect from sunlight	 Do not use if the packaging is damaged	 Store in a dry place
---	---	---	--	---	---	---	--	--

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BB med. product GmbH
Kroha GmbH