Please read instructions for use carefully before using the CANNASEN® CBD Nasal Spray Night.

How does CANNASEN® CBD Nasal Spray Night work?

The medical device for intranasal use:

- Moisturizes, cares for, and cleanses dry and crusted nasal mucous membranes.
- Facilitates breathing and thus ensures better sleep.

CANNASEN® CBD Nasal Spray Night is a medical device. Efficacy and safety have been tested according to the European Directive for Medical Devices (93/42/EEC).

Active Principle of CANNASEN® CBD Nasal Spray Night

CANNASEN® CBD Nasal Spray Night was developed especially for night-time usage. CANNASEN® CBD Nasal Spray Night moisturizes the mucous membrane without impairing the function of the cilia. In addition, the product loosens the crusts and protects against chronic inflammation. This makes breathing easier. Dry winter air as well as heated and air-conditioned rooms can quickly dry out the nasal mucous membranes and weaken the natural protective function of the nose. The nasal mucous membranes become cracked and brittle, swell up, and make it difficult to breathe through the nose. For these reasons, CANNASEN® CBD Nasal Spray Night was developed to improve sleep quality.

Application areas

For moistening of the nasal mucosa.

Dosage instructions, type and duration of application

Open the bottle by removing the cap. When using the spray for the first time, pump a few times until the oil comes out in a fine spray jet.

Spray onto a piece of paper or a tissue to prevent the oil from dripping onto clothing or the surrounding area. After that, the spray is ready for all further applications. Carefully insert the spray head into one nostril. Pump 1–2 times (see dosing instructions) into each nostril while inhaling through the nose.

Hold the bottle upright and vertically. The bottle should not be held face down.

Allow the product to act briefly. Then blow your nose if necessary or simply wipe off.

CANNASEN® CBD Nasal Spray Night can be used once a day in the evening before going to bed. The product is intended for temporary use.

General precautions

- Avoid contact with eyes.
- Not recommended for children and adolescents under 14 years of age. Use only after consulting your doctor.
- Keep out of reach of small children.

- Keep out of reach of pets.

- Persons with an allergy to Sesame Oil or any of the ingredients should not use this product.

- Use during pregnancy or breastfeeding only after consulting a doctor.

- For hygiene reasons, the product should be used by only one person. The applicator should be cleaned with a clean cloth after each use.

Contraindications

CANNASEN® CBD Nasal Spray Night should not be used in cases of known hypersensitivity to any of the ingredients. Please do not use the product at the same time as other products for the nose.

Interactions

In order to avoid interactions with medicines, apply medication two hours before or after use or consult your doctor.

Reporting of incidents

If this product is associated with a serious incident (death of a patient, user, or other person; temporary or permanent serious deterioration in the state of health of a patient, user, or other person; serious risk to public health), this incident must be reported to the manufacturer and the National competent authorities.

Notes and information on the shelf life of the medical device

CANNASEN® CBD Nasal Spray Night may be used only until the date marked “use by” on the folding box.

Use within 3 months after opening.

Due to the nature of the ingredients, a slight darkening may occur after some time. This does not affect the quality of the product.

Dosage form and content

Original packaging with 15 ml spray.

Ingredients

Active ingredients: Sesame Oil.

Other ingredients: Hemp Oil with Cannabinol (CBN) and Cannabidiol (CBD), Vitamin E, Lavender Oil.

Storage

Store in a cool place (between 10°C and 25°C) away from light.

Disposal

The product can be disposed of with normal household waste.

General information:

Poor breathing is one of the main causes of poor-quality sleep. This can lead to difficulty concentrating in everyday life. CANNASEN® CBD Nasal Spray Night acts like a massage for the nose, thereby improving breathing – a function that can help to improve night's sleep.

CANNASEN® CBD
THE MEDICAL BRAND
DENMARK

Manufacturer:
CanNordic A/S
Fruebjergvej 3
2100 Copenhagen Ø
Denmark

Symbol CE

Upper /
lower temperature limit

Use by

Please observe
the instructions
for use

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