Dear customer,
Thank you for choosing Actimove® Rhizo Forte.

Product description
Actimove® Rhizo Forte is a thumb orthosis for the treatment of painful conditions and irritations of the thumb joints associated with chronic or acute indications. The orthosis immobilizes the carpometacarpal (CMC) and the first metacarpophalangeal (MCP 1) joints and places the thumb in a functional position. Its special construction effectively restricts thumb adduction, while leaving the four fingers and the wrist to move freely. Its adaptable aluminum core makes it easy to optimally fit the immobilizer to the individual hand shape. With its fasteners, the orthosis can be easily secured with one hand. The selected materials and the low-profile design with smooth edges contribute to maximum comfort. The orthosis is dirt resistant, can be used in water and even when in contact with food, and can be dried easily. Please follow the instructions below!

Intended purpose
Actimove® Rhizo Forte is intended to provide immobilization of the thumb and used in the treatment of painful conditions and irritations of the thumb joints including

- Thumb osteoarthritis
- Rheumatoid arthritis of the thumb
- Ulnar collateral ligament injuries of the MCP 1 joint (e.g. skier's thumb)
- Instability of the CMC joint
- Soft tissue injuries (sprains)
- Traumatic thumb dislocation
- Ligament instabilities
- Postsurgical and nonsurgical treatment.

Please contact your healthcare professional if you are unsure whether the product is appropriate for you.

Directions for use
Actimove® Rhizo Forte is a moldable anatomically pre-shaped orthosis intended for single patient use. For adequate fitting and defining the duration of the application we recommend having Actimove® Rhizo Forte applied and adjusted by a healthcare professional the first time you wear it.

Application

1. Pull Actimove® Rhizo Forte over your hand like a glove, positioning your thumb in the support as indicated.
2. Close the strap around the thumb.
3. Then close the straps around the hand.
4. The orthosis can be molded on the individual hand shape for best fit.

Note: Do not apply with tension.

Cleansing and care
Actimove® Rhizo Forte is to be cleaned by hand wash using a mild detergent. To retain the shape of your orthosis please dry flat. Do not bleach and dry-clean or iron. Please close all straps prior to washing.

Notice
Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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