EN

This medical device and Instructions for Use are intended for use by a lay person.

Read the instruction for use before starting the treatment with Sorbact® Fungi.

Device Description

Sorbact® Fungi is a DACC™-coated bacteria and fungi binding ribbon gauze, based on Sorbact® Technology. Sorbact® Fungi treats fungal infections in skin folds (intertrigo).

Intended Purpose

Sorbact® Fungi is intended to treat fungal infections in skin folds (intertrigo), e.g. abdominal folds, between fingers and toes, under the breasts and in the groin.

Who can use Sorbact® Fungi?

Sorbact® Fungi is intended to be used on children and adults. The dressing can be used by healthcare professionals and lay persons in healthcare facilities and home environment.

How does Sorbact® Fungi work?

Sorbact® Fungi binds common microorganisms, such as *Staphylococcus aureus* (including MRSA), *Streptococcus* spp., *Escherichia coli*, *Pseudomonas aeruginosa* and *Candida albicans*, as shown *in vitro*. There is no release of antimicrobial agents.

When shall Sorbact® Fungi not be used?

There are no known contraindications to the use of Sorbact® Fungi. Warnings and precautions should be noted.

Warnings and Precautions

Do not use if you have previously experienced hypersensitivity to the dressing (dialkylcarbamoyl chloride [DACC™], cotton).

Do not use fatty products such as lotions or antifungal creams/ointments in combination with the dressing as they may decrease the binding of microorganisms.

Do not re-use. Sorbact® Fungi is for single patient and single use only. Re-use may lead to lack of effect. Do not wrap around extremities such as fingers or toes, as it may impair circulation.

Consult a healthcare professional if infection fails to improve or deteriorates (extended area, redness, odor).

How to use Sorbact® Fungi

- Clean and dry the affected area.
- Cut the dressing with a clean scissor to a suitable length. Apply the dressing. Ensure that the dressing comes into direct contact with the whole affected area to allow microorganisms to bind to the dressing.
- For fixation, underwear or a skin-friendly adhesive can be used. The adhesive should be applied outside the affected area.
- Daily dressing change in conjunction with inspection and regular hygiene routines is recommended.
- Continue to use the dressing for 2-3 days after the symptoms have resolved to avoid recurrence.

Storage and Disposal

Sorbact® Fungi shall be stored dry and kept away from sunlight.

Disposal should be made according to local environmental procedures.

Notice to User

Any serious incident that has occurred in relation to the device should be reported to ABIGO Medical AB (complaints@abigo.com) and the competent authority of your state.

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Sorbact® is a registered trademark of ABIGO Medical AB
ABIGO Medical AB, Vapenvägen 1, SE-696 33 Askersund, Sweden + 46 31 748 49 50
SE ABIGO Medical AB, Vapenvägen 1, SE-696 33 Askersund, Tel. + 46 31 748 49 50, sorbact.se
FI Oy Verman Ab, PL 146, 04201 Kerava/Kervo info@verman.fi
DK ABIGO Pharma A/S, Stengårdsvej 25, 4340 Tølløse, Tlf. +45 4649 8676, sorbact.dk