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EN

Patient information leaflet: DEXERYL® cream

MD Medical Device

What DEXERYL® cream contains

Glycerol, white soft paraffin, liquid paraffin,
glycerol monostearate, stearic acid, dimethicone,
macrogol 600, trolamin, acrylamide/acryloyldi-
methyl taurate copolymer, isohexadecane,
polysorbate, pentylene glycol, ethylhexylglycerin,
carbomer, purified water.
Tubes of 250 g and 50 g.

What DEXERYL® cream is used for

- As a skin protection cream for the treatment of
signs and symptoms of skin dryness (xerosis)
especially in various skin diseases (such as
atopic dermatitis, Ichthyosis...) and erythema
induced by radiotherapy. From birth.

How DEXERYL® cream acts on the skin

DEXERYL® cream is formulated for the treatment
of signs and symptoms of skin dryness (xerosis):
- The filmogenic effect of white soft paraffin and
liquid paraffin which:

- physically inhibits evaporation thus preventing
dehydration (water loss),
- has a barrier effect against external damage;

NO

Pasientinformasjon : DEXERYL® krem

MD Medisinsk utstyr.

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Glycerol, white soft paraffin, liquid paraffin,
glycerol monostearate, stearic acid, dimethicone,

- In addition, the hydrating effect of glycerol helps
to maintain the balance of moisture in the skin.
Thanks to its formulation, DEXERYL® cream also
reduces and helps to prevent irritation.

How to use DEXERYL® cream

Cream for external use only. Apply a thin layer
of cream to the skin area to be treated once or
twice daily, or more frequently if necessary but
never more than 6 times daily.

When not to use DEXERYL® cream

Do not apply if you are allergic to any of the
ingredients.

Considering the lack of relevant information, it is
preferable not to use the cream in conjunction
with other topically applied products. If in any
doubt, ask a health professional for advice.

Warnings and special instructions for use

Do not apply on wounds, mucous membranes
and eyes.

Do not swallow.

Keep out of the reach of children.

Tell your health professional about any undesirable
side effects.

If you have reason to believe that a serious
incident occurred linked to the use of this product,
please report it to your national authority and to
the manufacturer.

Special precautions for storage:

Do not store above 30 °C. Do not use after the
expiry date shown on the tube. Once opened, use
within 6 months.

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