# Mona Lisa

## Neo-Safe®

### T CU 380

### T CU 380 Mini

Intrauterine Contraceptive Device  
Intrauterinpressar zur Empfängnisverhütung  
Dispositivo Contraccettivo Intrauterino  
Intrauterint præventionsmiddel

Information for the user  
Informationen für die Benutzerin  
Informazioni per la paziente  
Information til kvinder

(en)

### Mona Lisa Neo-Safe®

T CU 380

T CU 380 Mini

Intrauterine Contraceptive Device

## INFORMATION FOR THE USER

## Design and dimensions

The Neo-Safe® T CU 380/380 Mini Intrauterine Device (IUD) is made of a T-shaped frame of polyethylene and barium sulphate. The vertical arm is wound with copper wire. The copper surface is 380 mm<sup>2</sup>. Two nylon threads are attached to the lower end of the vertical arm.

## Dimension:

Neo-Safe® T CU 380: 31.8 mm wide and 31.9 mm long.  
Neo-Safe® T CU 380 Mini: 24.0 mm wide and 30.0 mm long.  
The polyethylene frame and the wire are radio-opaque.

## Mode of action and possible effects

There are different types of intrauterine contraceptive devices. Wide-ranging international studies still verify that Copper-IUDs are not only safe but also effective over long periods. In addition, it is also generally recognized that Neo-Safe® T CU 380/380 Mini IUDs are one of the most effective means of contraception. The Neo-Safe® T CU 380/380 Mini is shaped in such a way as to prevent irritation of the uterus. It is usually inserted during menstruation. The insertion is normally painless. The opening of the uterus is examined, the length of the uterine cavity is determined and the Neo-Safe® T CU 380/380 Mini is carefully inserted into the uterine cavity, until it is in the correct position and touches the fundus. As soon as the Neo-Safe® T CU 380/380 Mini has been inserted, you are protected against pregnancy.

Neo-Safe® T CU 380/380 Mini is effective for 5 years. IUDs are highly effective and can only be compared with the oral contraceptive pill for the prevention of pregnancy. No contraceptive method offers 100% safety. However, in practical terms, with a Neo-Safe® T CU 380/380 Mini you are protected.

It is not fully known how IUDs work. However, nowadays it is assumed that the most probable effect is that it disrupts the normal function of the male gametes (sperm), which become incapable of fertilizing the female egg. It is also assumed that copper ions influence the development of the egg so that fertilization does not take place. IUDs are no longer considered a method of abortion.

## Contraindications

Neo-Safe® T CU 380/380 Mini should not be used in the case of:

1. Malignant diseases of the genital tract
2. Vaginal bleeding
3. Pregnancy
4. Infections of the genital tract
5. Sexually transmitted diseases during the last 12 months (except bacterial vaginitis, repeated herpes infection, hepatitis B)
6. Abortion with infection during the last 3 months, pelvic inflammatory disease (PID)
7. Uterine malformations (congenital or acquired)
8. Allergy to copper

The application should be reconsidered in:

1. Valvular heart disease
2. Anaemia
3. Coagulation disorders
4. Anti-inflammatory treatment
5. Wilson's disease
6. Multiple sexual partners

## Possible complications and precautions

The first three cycles following insertion are usually different from your normal cycles. Periods can be heavier, cramps and spotting may occur both before and afterwards. Three to four cycles after insertion of the IUD, menstruation normally

resumes as before insertion. Sometimes these changes are experienced between the menstrual periods during the cycle. The absolute risk of ectopic pregnancy is extremely low due to the high effectiveness of intrauterine devices. However, when a woman becomes pregnant during intrauterine contraceptive use, the relative likelihood of ectopic pregnancy is greatly increased, and should be excluded. Due to the risk of infections and a possible higher risk of expulsion, the insertion of an IUD in nulliparous women may be reconsidered and the advantages and possible therapeutic risks should mutually be weight up carefully.

You should contact your doctor if:

- you cannot feel the threads of the IUD,
- you or your partner are able to feel the lower end of the IUD,
- you think you are pregnant,
- you experience abdominal pain, fever or unusual vaginal discharge,
- you or your partner feel pain or discomfort during sexual intercourse,
- your menstruation is delayed, since rarely a pregnancy outside the uterine cavity, e.g. in a fallopian tube (ectopic pregnancy) may occur,
- you wish to have the IUD removed and / or want to become pregnant.

Cases of expulsion of the IUD have been reported in women using a menstrual cup but there is no certainty whether the expulsion is caused by the cup. There is a possibility of a suction effect on the IUD when the menstrual cup is removed incorrectly. Please ask your doctor for advise on the correct use of the menstruation cup.

## Insertion and removal

Before the insertion, you must be informed on the efficacy, risks and side effects of Neo-Safe® T CU 380/380 Mini. A gynaecological examination including pelvic examination and a cervical smear should be performed. Pregnancy, genital infections and sexually transmitted diseases should be excluded. The position of the uterus and the size of the uterine cavity should be determined.

The instructions for insertion should be carefully taken into account and the patient card be filled out together with the attending doctor. You should return for check-up 4 to 12 weeks after insertion and once a year thereafter or more frequently if clinically indicated.

Insertion is recommended during or shortly following menstruation. If pregnancy is excluded, Neo-Safe® T CU 380/380 Mini may be inserted at any time of the cycle. It can be also inserted immediately after first trimester abortion. After having a baby, insertion should be postponed until six weeks after delivery, as this is associated with greater rates of perforation and expulsion. There are no contraindications to breastfeeding with an IUD in situ, but the risk of perforation for insertion while breastfeeding is increased.

Neo-Safe® T CU 380/380 Mini is removed by gently pulling on the threads. If the threads are not visible, and the device is in the uterine cavity, removal should be postponed until after the next menstrual bleeding since the threads usually become visible immediately after menstruation. If they are still not visible, the device may be removed using a narrow forceps. This may require dilatation of the cervical canal. After removal, pregnancy is immediately possible.

If you wish to continue using the method, a new device can be inserted at the same time. If pregnancy is not desired, the removal should be carried out during the menstruation.

Insertion and removal may be associated with some pain and bleedings. Occasionally, the procedure may precipitate fainting or a seizure if you suffer from epilepsy.

It is recommended that you visit your doctor for a check-up one month after Neo-Safe® T CU 380/380 Mini was inserted to make sure the IUD is still in your uterus and is positioned correctly. An annual visit is recommended thereafter. You should also check to make sure that the Neo-Safe® T CU 380/380 Mini is still in your uterus by reaching up to the top

of your vagina, while in a squatting position, with clean fingers to feel the threads. Do not pull on the threads.

If you can **not** feel the threads or you can feel more of the Neo-Safe® T CU 380/380 Mini than just the threads, consult your doctor immediately. This suggests that your Neo-Safe® T CU 380/380 Mini is not in the uterus or is incorrectly positioned which would increase your chances of getting pregnant. It is recommended that you use an additional birth control method until you have seen your doctor. It is good practice to check that Neo-Safe® T CU 380/380 Mini is in place once a month.

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