**Mabs Supportive Braces
Knee Strap
SE-NO-DK-FI**

karo

Reason for change: Relaunch of Mabs Support
Correction date: 24-02-2025
Artworker: Sissan

Proof version
3

Print version
X

COM code: COM-7754-01
Artwork No: 2005001_1
Previous Artwork No: -
SKU No: 1003311
CMO: -
CMO code: -
EAN No: -
Pharma No: -
Component Type: Leaflet
Language: English, Swedish, Norwegian, Danish, Finnish

Dimensions (WxH): 210 x 297 mm
Material: -
Machine Drawing ID: -
Font: -
Minimum Point Size: -
Print Process: Offset printing

Print Colours: Colourmatch to Pantone Matching System.

Technical (Non Printing).

OVERPRINT PREVIEW

NO

Karo Healthcare AB owns this artwork and is not responsible for any changes made by the CMO/printing house. Changes must be communicated with us and we will supply a new print PDF file. Both CMO and printing house are responsible for the printed end result to match the intended colours.

UK General information

This Mabs product has been designed to support your knee, follow the instructions below. Always consult a doctor or therapist if your problem has not improved within a week. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse or if you experience discomfort when using the product, discontinue the use and contact a doctor. Follow the washing instructions on the product label and package. If you have material allergies, always use caution the first time wearing the product and be observant of skin reactions. You can find the material composition on the product label.

Instruction

The product has been designed to support your knee and the patella tendon with a pad integrated in the strap. The pad should be positioned just under the kneecap, over the tendon. Attach the strap around the knee and close the hook and loop closure. You might have to adjust the pressure needed.

For more information, visit www.mabs.se

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