**Mabs Supportive Braces  
Universal Wrist Black  
SE-NO-DK-FI**

**karo**

Reason for change: Relaunch of Mabs Support  
Correction date: 05-03-2025  
Artworker: Sissan

Proof version  
**5**

Print version  
**X**


COM code: COM-7756-01  
Artwork No: 2005003\_1  
Previous Artwork No: -  
SKU No: 1003312  
CMO: -  
CMO code: -  
EAN No: -  
Pharma No: -  
Component Type: Leaflet  
Language: English, Swedish, Norwegian, Danish, Finnish

Dimensions (WxHxD): 210 x 297 mm  
Material: -  
Machine Drawing ID: -  
Font: -  
Minimum Point Size: -  
Print Process: Offset printing

Print Colours: Colourmatch to Pantone Matching System.

Technical (Non Printing).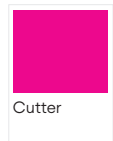

**OVERPRINT PREVIEW** **NO**

Karo Healthcare AB owns this artwork and is not responsible for any changes made by the CMO/printing house. Changes must be communicated with us and we will supply a new print PDF file. Both CMO and printing house are responsible for the printed end result to match the intended colours.

### **UK General information**

This Mabs product has been designed to support your wrist, follow the instructions below. Always consult a doctor or therapist if your problem has not improved within a week. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse or if you experience discomfort when using the product, discontinue the use and contact a doctor. Follow the washing instructions on the product label and package. If you have material allergies, always use caution the first time wearing the product and be observant of skin reactions. You can find the material composition on the product label.

### **Instruction**

This model can be used for both left or right by switching the splint position in the pocket. The default splint position from the factory is for the right hand. To switch the splint, remove it from the pocket and insert it into the other pocket. Open all the hook straps, slide it over the wrist and position the thumb in the hole. Close the two straps over the main body, attaching the front hook last. If needed, trim the length of the hook straps.

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