**(NL) Actimove®**

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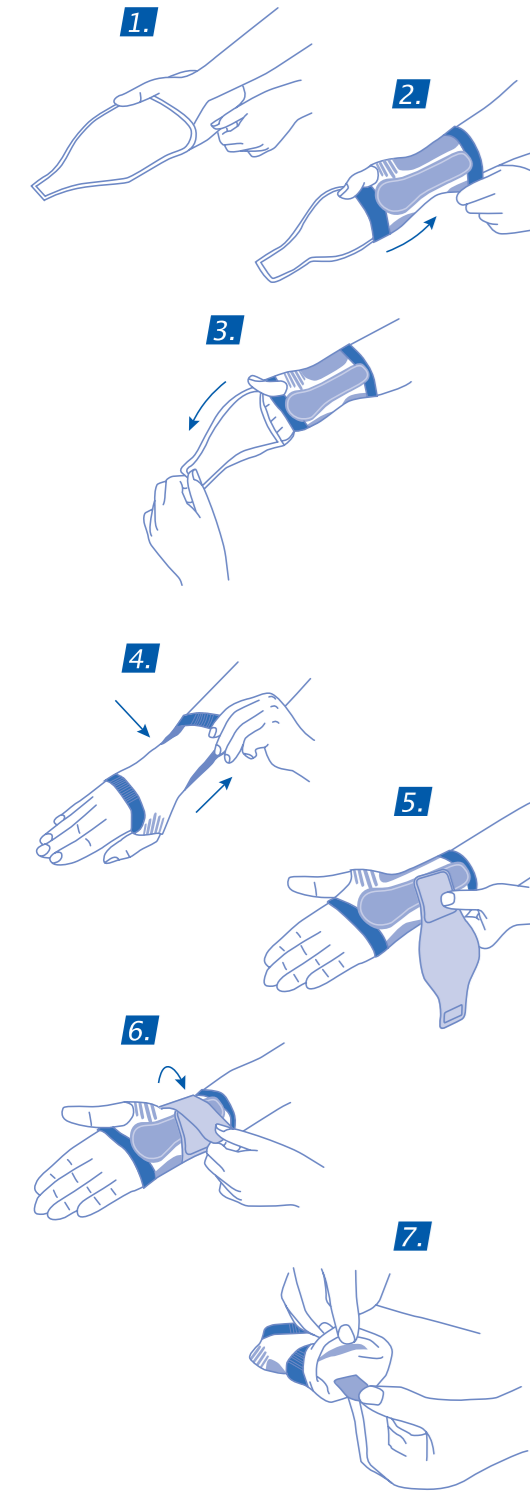
(EN) **Actimove®**

Dear Customer,  
Thank you for choosing Actimove® ManuMotion.

**Product description**  
Actimove® ManuMotion is an anatomically shaped wrist support specially designed for compression and stabilisation of the wrist. It combines advanced knitting structure with high-quality, latex-free materials for greater breathability and enduring comfort. The pressure pad on the backhand side provides an intermittent massage effect on the soft tissues intended to support the reduction of edemas and swelling. An anatomically shaped stay on the inner side of the hand stabilises the wrist in a functionally correct position, supporting pain relief. The Motion Comfort Zone in the thumb opening area provides a soft, smooth touch. A removable, inelastic wrist strap allows an additional, individually adjustable stabilisation of the wrist.  
Please follow the instructions below and get well soon!

**Intended purpose**  
Actimove® ManuMotion is intended for wrist support.  
Application fields include:

- Painful arthrosis
- Rheumatoid arthritis of the wrist
- Distortions (sprains)
- Tendovaginitis
- Tendon injuries
- Ligament injuries
- Soft tissue injuries
- Posttraumatic irritations
- Postoperative use

Please contact your healthcare professional if you are unsure whether the product is appropriate for you.

**Application**

1. Slide the donning aid over your hand, with the short side aligning with your palm. Leave your thumb outside.
2. Pull Actimove® ManuMotion over your hand and donning aid, and make sure that your thumb is located correctly.
3. Pull at the tapered end of the donning aid to remove it from underneath the support.
4. Pull the support over your hand and make sure that your thumb is located correctly.
5. The support is positioned correctly when the opening of the pressure pad is positioned right over the protuberance of your wrist.
6. For additional stabilisation apply the wrist strap by attaching the grey hook tab to the cover of the stay. Then pull the strap around your wrist and close the hook and loop closure.
7. To remove the stay (e.g. before washing) turn the lower part of your Actimove® ManuMotion inside out and take the stay from its pocket. When reinserting the stay, please make sure to insert the stay in the right direction.

For optimised wearing comfort Actimove® ManuMotion can be fitted by a healthcare professional or trained specialist. In accordance with your physician the bandage can also be worn without stay and wrist strap.

**Modification instruction**

The stay can be individually moulded at temperatures about 200°C. Moulding must be done by a technical professional.

**Cleaning and care**

Actimove® ManuMotion is hand or machine washable (using the gentle wash cycle at 30°C/85°F and a mild detergent). Please do not bleach, dry-clean, tumble dry or iron your wrist support and allow it to dry completely before wearing it again. Attention: Please make sure to remove the stay and to close the wrist strap before washing!

**Notice**

Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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