
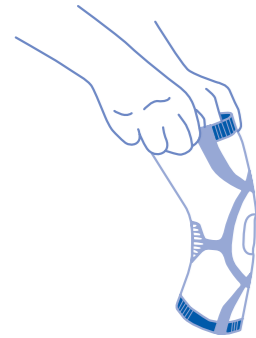
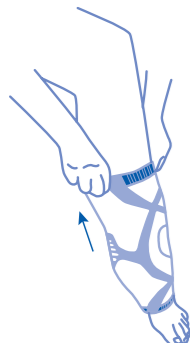
1.


2.

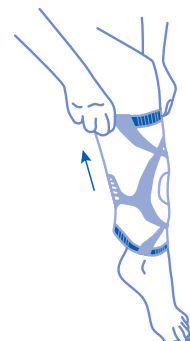3.

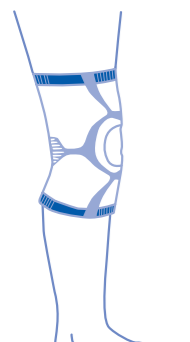4.

(NL)

## Actimove®

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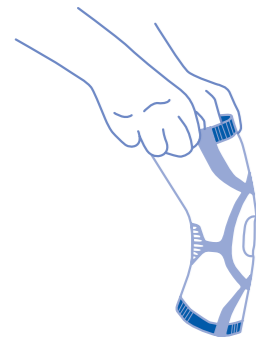
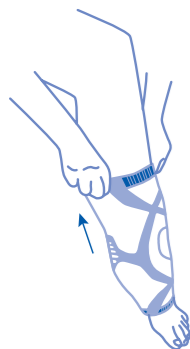
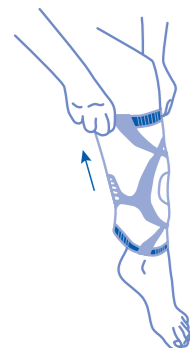
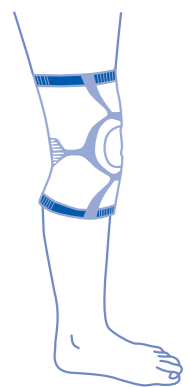
EN

Actimove®

Dear Customer,  
Thank you for choosing Actimove® GenuMotion.

### Product description

This product has been thoughtfully designed and developed together with orthopaedic specialists with great attention to detail to meet your expectations. Actimove® GenuMotion is a dynamic knee support featuring 3D knitting structure technology. It provides gradient, circular compression for stability and aids pain relief. The high quality, latex-free materials are soft and skin friendly, offering a breathable alternative to neoprene. The anatomically shaped pressure pad creates an intermittent massage effect to the surrounding tissue, intended to support blood flow and the natural outflow of lymph. This effect is important for the reduction of swelling and contributes to the healing process. Please follow the instructions below and get well soon!

### Intended purpose

Actimove® GenuMotion is intended for knee support. Application fields include: Actimove® GenuMotion is indicated for gonarthrosis and arthritis, chondromalacia with or without recurrent joint swelling, minor meniscal damages, fibromyalgia (tendomyopathy) and minor instabilities of the knee joint. It is also recommended for posttraumatic irritations and postoperative use. Please contact your healthcare professional if you are unsure whether the product is appropriate for you.

### Application

Slip into your Actimove® GenuMotion until the stabilising pressure pad surrounds your patella. Adjust the knee support until the fit is snug and comfortable (see pictures 1–4 for details).

### Cleansing and care

Actimove® GenuMotion is hand or machine washable (using the gentle wash cycle at 30°C/85°F and a mild detergent). To retain the shape and medical compression of your knee support please do not bleach, dry-clean, tumble dry or iron.

### Notice

Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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