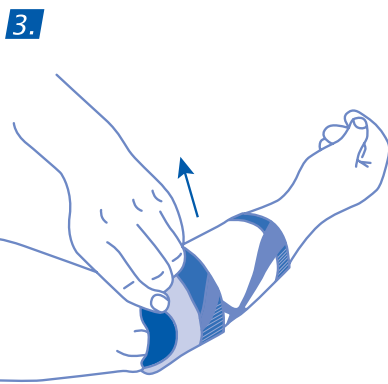
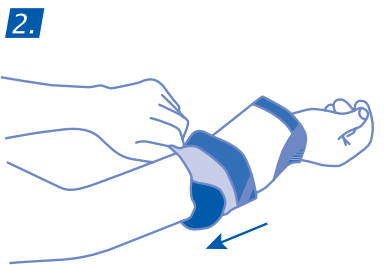
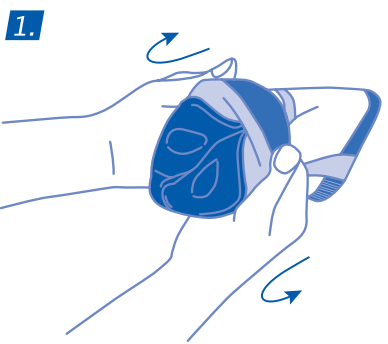
NL Actimove®

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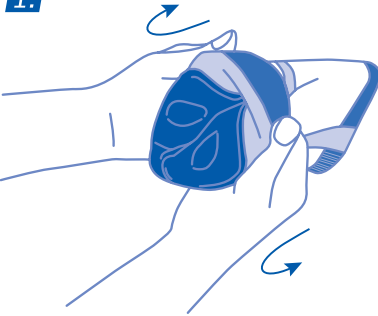
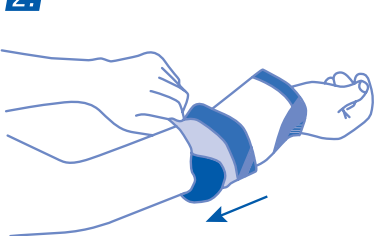
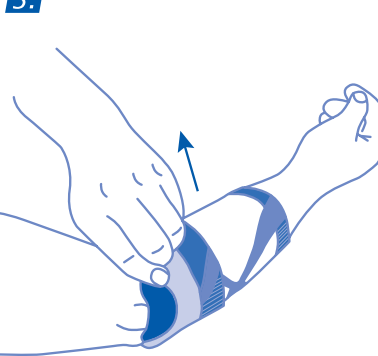
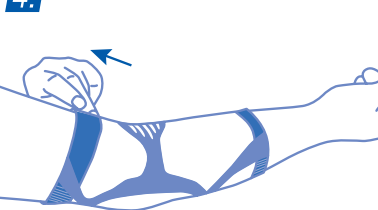
EN

Actimove®

Dear Customer
Thank you for choosing Actimove® EpiMotion.

Product description

Actimove® EpiMotion is an elbow support combining 3D knitting technology for optimised compression and stability with high-quality, latex-free materials for high breathability, softness and enduring comfort. Two anatomically shaped pelottes gently massage the elbow area for pain relief. The Actimove® EpiMotion support features a special designed two-way stretch knitted material for a secure fit during activity. Please follow the instructions below and get well soon!

Intended purpose

Actimove® EpiMotion is intended for elbow support. Application fields include:

- Tennis elbow (lateral epicondylitis)
- Golfers elbow (medial epicondylitis)
- arthritis (joint inflammation)
- arthrosis
- rheumatoid joint effusion
- bursitis olecrani (student's elbow)
- insertion tendopathy of the triceps brachii
- posttraumatic irritations
- postoperative use

Please contact your healthcare professional if you are unsure whether the product is appropriate for you.

Application

1. Turn the upper part of your Actimove® EpiMotion inside out until the circular cut-outs in the two pelottes are visible (Fig. 1).
2. Pull the bandage onto your arm until the pelottes are right below the elbow joint, with the outer pelotte pointing towards the bony prominence on the outside of your elbow (Figs. 2 and 3).
3. Unroll the upper section over the elbow, ensuring that the bony prominence on the outside of your elbow is in the circular cut-out of the outer pelotte (Fig. 4).
4. The bony prominence on the inside of your elbow should be located in the circular cut-out of the inner pelotte.

For further application advice a healthcare professional or trained specialist can be consulted. Actimove® EpiMotion is intended for single-patient use.

Cleansing and care

Actimove® EpiMotion is hand or machine washable (using the gentle wash cycle at 30°C/85°F). To retain the shape and medical compression of your elbow support, please do not bleach, dry-clean, tumble dry or iron.

Notice

Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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