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Step 1

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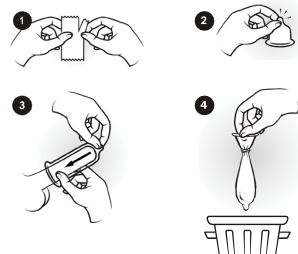
EN Instructions for use:

Step 1

Open package carefully and remove the condom. Make sure you avoid damaging the condom with your nails, jewellery or anything else. The condom must be on before intercourse begins, and it must remain on throughout intercourse in order to prevent sexually transmitted diseases or unwanted pregnancy.

Step 2

First look at the condom, so you are certain it is facing the right way. Place the rolled condom on the tip of the penis and make sure to pinch the air out of the tip of the condom by squeezing it between the thumb and first finger.

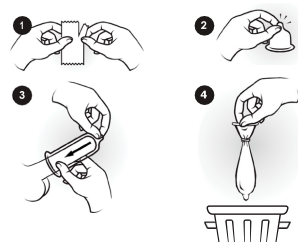
Step 3

While still holding the tip of the condom, roll the condom down over the penis. Then check that the condom is seated correctly.

Step 4

Shortly after ejaculation, the condom must be held firmly in place at the base of the penis before it is withdrawn. Then remove the used condom and dispose of it hygienically. Never flush condoms down the toilet.

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Please read before use:

CE CE marking. CE 2460.

Read the package leaflet with instructions for use.

The condom can only be used once, otherwise there is a risk of infection and pregnancy.

The condom is made from natural rubber latex. Condoms made from natural rubber latex may cause allergic reactions, including anaphylactic shock, in individuals hypersensitive to latex.

Always store condoms in a dry place away from direct sunlight and heat.

Never use the condom if the box is damaged.

Always check the expiry date. See back of packaging.

Manufacturer. See back of packaging.

Lot number. See back of packaging.

If you wish to use a lubricant, use one specifically designed for condoms. Never use oil-based creams, butter, margarine or massage oil, as these may damage the condom.

If unsure, ask at the pharmacy. If any form of medication is being used in the vagina or on the penis, first ask your doctor for advice.

If the condom breaks or falls off during intercourse, withdraw immediately and avoid ejaculation.

If ejaculation has already taken place, consult a doctor as soon as possible and within 72 hours.

All of our condoms meet international quality standard ISO 4074:2015.

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